



INDEX

DK	Planløftebøjle BS, 150 kg	2
GB/US	Horizontal lifter BS, 150 kg (330 lbs)	8
DE	Horizontaler Liftbügel BS, 150 kg	15
SE	Planlyft BS, 150 kg	21
FR	Cintre horizontal Body Scoop, 150 kg	27
IT	Sollevatore orizzontale BS, 150 kg	33
ES	Elevador horizontal BS, 150 kg	39

Vers. 2.00

CE

Guldmann[™]

GB_US. . . Horizontal lifter BS, 150 kg (330 lbs)

Vers. 2.00

Item nos:
28425

1.00	Purpose and use	9
1.01	Manufacturer	9
1.02	Purpose	9
1.03	Important/warnings	9
2.00	Commisioning and functional description	10
2.01	Unpacking and Preparation	10
2.02	Labels	10
2.03	Functional description	10
3.00	Maintenance	12
3.01	Cleaning and Maintenance	12
3.02	Replacement of components	12
3.03	Service life	12
3.04	Storage and use environment	12
4.00	Technical specifications	12
5.00	EC-Declaration of conformity	12
6.00	Environmental policy statement - V. Guldmann A/S	13
A.	Users guide	14
B.	WARRANTY	14

1.00 Purpose and use

1.01 Manufacturer

V. Guldmann A/S
Graham Bells Vej 21-23A
DK-8200 Aarhus N
Tel. + 45 8741 3100
Fax + 45 8741 3131
www.guldmann.com

1.02 Purpose

The Horizontal lifter BS is a lifting device which is used together with a Guldmann Ceiling hoist and rail system to move a deceased person in a horizontal position. The Horizontal lifter BS may be used only for horizontal relocation^{*)} of a deceased person lying on a level, horizontal surface. A level surface can be, e.g. a bed, a stretcher, a casket or the floor.

Conditions for use

The use of the Horizontal lifter BS is subject to the following:

- The Horizontal lifter BS is used by trained personnel.
- The user pays attention to secure safe transfer with the Horizontal lifter BS.
- The Horizontal lifter BS may be used only with a hoist and rail systems which have been tested and approved according to Guldmann's instructions.

**) A horizontal relocation is defined as a relocation of a deceased person lying on a horizontal surface where the horizontal position is kept during a vertical, horizontal or combined movement. The surface can be completely level or at an angle suited to the respective situation.*

1.03 Important/warnings

Always exercise caution when using the Horizontal lifter BS. This means that the user should always make sure that he/she and any other persons are safe in order to avoid injuries or accidents.

- Please read the user manual before starting to use the Horizontal lifter BS.
- Do not exceed the recommended safe working load of the Horizontal lifter BS.
- Products marked with the lowest lifting capacity determine the maximum load of the whole system.
- Use the Horizontal lifter BS only to move a deceased person in a horizontal position.
- If a defect appears during the use of the Horizontal lifter BS, stop using it and contact the Guldmann Service Team for repair.
- To avoid unintentional movements of the Horizontal lifter BS, the user must position the hoist and the Horizontal lifter BS centrally over the deceased person before lifting is started.

2.00 Commissioning and functional description

2.01 Unpacking and Preparation

Visual check of the Horizontal lifter BS


If the Horizontal lifter BS is thought to be damaged upon receipt, it must not be used before it has been checked and approved by a qualified person or the Guldmann Service Team.

The contents of the box

- Horizontal lifter BS
- Horizontal lifter BS manual

2.02 Labels


ID number label

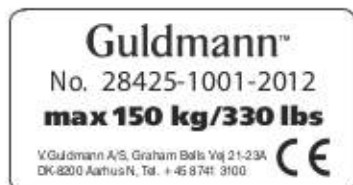

Injury warning (yellow)

2.03 Functional description

1. Hoist attachment point
2. Swivel link
3. Scissors
4. Scoop locking bush
5. Scoop

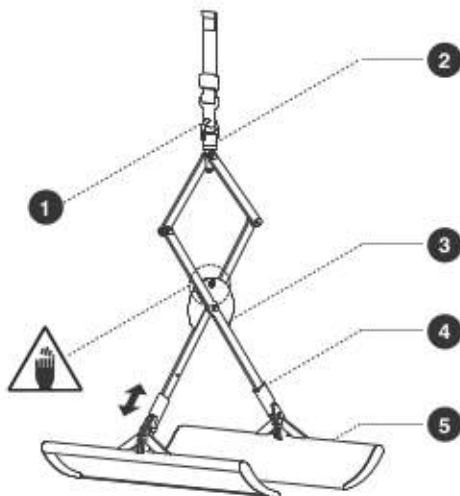

The scoop blades can be released by sliding locking bush up, and the scoop blades are locked by sliding locking bush down.




Warning

Beware of fingers being squeezed in scissors.


Attention should be paid to ensure safe transfer and the work technique of the user.

When transferring a deceased person the Horizontal lifter BS must be lifted over the deceased person and the scoop blades placed one on each side and aligned to the deceased person's center of gravity in the length direction.

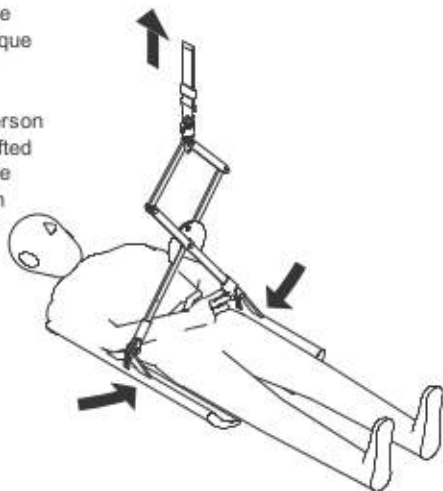

When activating the ceiling hoist, the scissors of the Horizontal lifter BS will create an inward pressure assisting the scoop blades to slide under the deceased person being transferred. Make sure that the locking bushes are securely closed.

The locking bushes can be released, to allow the scoop blades to swing open, when removing the Horizontal lifter BS from e.g. a casket.

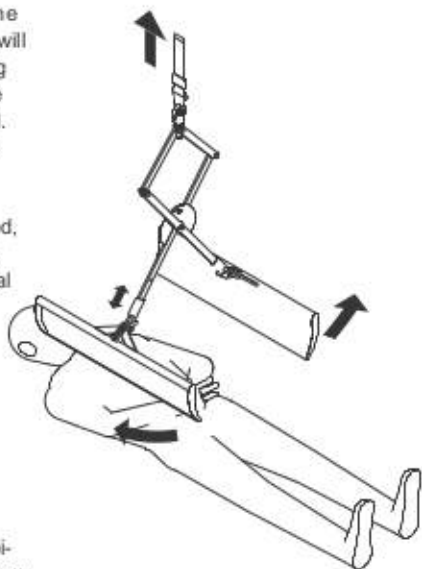

Guldmann disclaims any responsibility for defects or accidents which may occur as a result of incorrect use of the Horizontal lifter BS or because of a lack of attention on the part of the user.

3.00 Maintenance

3.01 Cleaning and Maintenance

The Horizontal lifter BS is made of stainless steel and resistant materials.

To ensure a proper appearance and long lifetime of the Horizontal lifter BS, it is recommended to follow these guidelines:

- Routinely cleaning of the Horizontal lifter BS
- Cleaning with Alcohol based detergents is recommended to obtain optimum hygiene.

The Horizontal lifter BS does not need any additional maintenance apart from general cleaning.

3.02 Replacement of components

Components may be replaced only by a qualified service technician or the Goldmann Service Team.

3.03 Service life

The Horizontal lifter BS has an expected service life of 10 years, when subjected to continuous maintenance

3.04 Storage and use environment

The Horizontal lifter BS is designed for indoor use.

Temperature -10 C° to 40 C°

Keep the Horizontal lifter BS in a dry room for long-term storage.

4.00 Technical specifications

Lifting capacity max 150 kg (330 lbs)

Weight 10.2 kg (22.5 lbs)

Dimensions

Horizontal lifting support (W x H x L) variable x 1560 to 1860 x 740 mm
(variable x 4'-5" x 2.5')

Materials

Horizontal lifting support Stainless steel

5.00 EC-Declaration of conformity

The products are manufactured in compliance with the Council Directive 93/42/EEC of 14 June 1993 – with amendments, as medical device class 1.

Guldmann is continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

It is Guldmann's goal to:

- Comply with the current environmental legislation (e.g. WEEE and REACH directives)
- Ensure that we, at the widest possible range, use RoHS compliant materials and components
- Ensure that our products do not have an unnecessary negative impact on the environment regarding use, recirculation or disposal
- Ensure that our products contribute to a positive working environment in the places they are utilised

Inspections are made annually by the Department for Nature and Environment from the Municipality of Aarhus using the Danish Environmental Protection Act, section 42 as a reference.

USA and countries outside the EU

- A. Users guide**
Before using the product, read the entire operation manual including warranty.

B. WARRANTY

Guldmann warrants its equipment is free from material defects under normal use, and will perform substantially in accordance with the specifications set forth in documentation provided with the equipment.

This express warranty shall be in effect for one year from the date of original purchase and installation (the "Warranty Period"). If a valid claim is made during the Warranty Period for malfunction or equipment defect, Guldmann will repair or replace the equipment at no additional cost to you. Guldmann retains sole discretion as to whether the equipment will be repaired or replaced.

This warranty shall be null and void if the equipment is operated and maintained in any manner inconsistent with its intended use or the instructions provided with the product. Further, in order for the warranty to remain in effect for the full Warranty Period, all service to the equipment must be provided by a Guldmann designated technician. Any parts or components repaired or replaced by a Guldmann designated technician will be guaranteed for the remainder of the Warranty Period.

The warranty does not cover any part of the equipment which has been subject to damage or abuse by the user or others. The warranty does not cover any part of the equipment which has been altered or changed in any way by the user or others. Guldmann does not warrant that the lifting device functions will meet your requirements, be uninterrupted or error free.

The warranty set forth is in lieu of all other express and implied warranties, whether oral, written or implied, and the remedies set forth above are your sole and exclusive remedies. Only an authorized officer of Guldmann may make modifications to this warranty, or additional warranties binding on Guldmann. Accordingly, additional statements such as advertising or presentations, whether oral or written, do not constitute warranties by Guldmann.

Service or Repair

Contact Guldmann Repair for an authorization to return any defective item during the Warranty Period. You will be provided with a return authorization number and address for returning the item for warranty service or replacement. Do not return items to Guldmann under warranty without receiving a Return Authorization Number.

If mailing the item, pack it carefully in a sturdy carton to prevent damage. Include your Return Authorization Number, a brief description of the problem and your return address and phone number. Guldmann does not assume the risk of loss or damage while in transit, so it is recommended you insure the package.

| Time to care |

V. Guldmann A/S

Head Office:
Graham Bells Vej 21-23A
DK-8200 Aarhus N
Tel. +45 8741 3100
Fax +45 8741 3131
info@guldmann.com
www.guldmann.com

Guldmann GmbH

Unter den Eichen 5
D-65195 Wiesbaden
Tel. +49 611 974 530
Fax +49 611 300 018
info@guldmann.de
www.guldmann.de

Guldmann Sverige AB

Smålandsgatan 4
S-441 57 Alingsås
Tel. +46 0322 55290
Fax +46 0322 55320
info@guldmann.se
www.guldmann.se

Guldmann Sarl

66 rue du Moulin de la Pointe
F-75013 Paris
Tél. +33 145 54 78 36
Fax +33 145 57 10 52
france@guldmann.com
www.guldmann.fr

Guldmann Srl

Via Focherini 7
I-43058 Sorbolo (PR)
N° Verde 800 781 604
Tel. +39 0521 660132
Fax +39 0521 691399
Cell. +39 340 9087107
italia@guldmann.com
www.guldmann.it

Guldmann Inc.

14401 McCormick Drive
Unit A
Tampa, FL 33626
Tel. 800 664 8834
Tel. 813 880 0619
Fax 813 880 9558
info@guldmann.net
www.guldmann.net