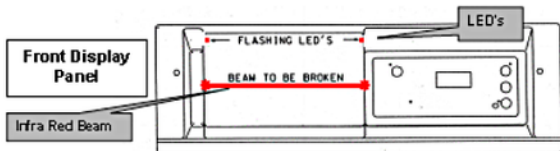


**MALMET ES BEDPAN FLUSHER SANITIZER****Quick Reference Guide**

For Complete Instructions please refer to the Operations, Maintenance and Installation Manual on this unit.


**OPERATING MACHINE**

- 1. OPEN DOOR – BREAK BEAM WITH CLOSED FIST or PAN/BOTTLE**
- 2. LOAD PAN & BOTTLES AS PER INSTRUCTIONS (Page 6)**
- 3. CLOSE DOOR & START CYCLE - BREAK BEAM WITH CLOSED FIST**
 - a. DOOR WILL CLOSE** Machine displays **rdY**
 - b. BREAK BEAM** within 8 seconds with **CLOSED FIST** and cycle will start.

If not broken in 8 seconds beam must be broken **TWICE** to reopen.

BEDPAN INSERTION

Refer Page 6 in the Operation, Maintenance and Installation Manual.

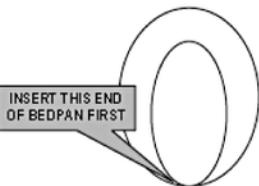**STANDARD SLIPPER PAN INSERTION**

NOTE: SLIPPER PAN RACK MUST BE INSERTED INTO THE MAIN RACK BEFORE INSERTING SLIPPER PAN.

Pan must be firmly pushed down and locked into rack.

Refer Pages 13 & 14 in the Operation, Maintenance and Installation Manual.

