



Hills Healthcare Equipment

SAFETY NOTICE

To prevent user injury or death please follow the safety instructions below:

- 1) Ensure the user is physically and cognitively able to use a bed pole/rail. The user must be physically and cognitively able to manoeuvre themselves with the assistance of the bed pole/rail.
- 2) Do not use the bed pole/rail if the user has a history of falling from bed.
- 3) The bed pole/rail should only be used in an environment where a carer is immediately nearby and available for assistance.
- 4) Position the base section of the bed pole/rail underneath the mattress so that it fits between the mattress and the bed base. Ensure there is no gap between the edge of the mattress and the upright section of the bed pole/rail.
- 5) Do not tie or fix the bed pole/rail to the bed.
- 6) Ensure sufficient weight is placed upon the bed pole/rail to limit movement of the product while the user is in bed.
- 7) On beds with rising backrests, position the bed pole/rail on the lower (foot) end section of the bed.
- 8) Do not use a bed pole/rail on beds with a profiling mattress frame.
- 9) Regularly check the position and stability of the bed pole/rail and frequently assess the user for their ability to use the product. Do not use this product for users with limited mobility or cognitive impairment.

Should you require further advice, please contact our Customer Service team.

Customer Service Team contact details:

Tel: 08 92484444

Email: customerservice@hillshealthcare.com.au

 **Evocare**
www.evocare.com.au
Ph: 07 3355 8000 Fax: 07 3355 5043