

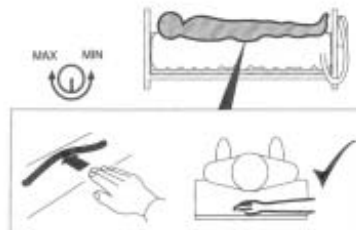
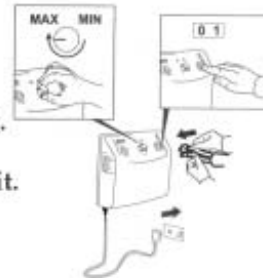




BlueWave II Mattress Replacement

Quick Set Up Guide

Note: Allow up to 30 minutes to inflate completely
Do not place the patient on the system during inflation

- 1 Remove existing mattress, covers and sheets from bed
- 2 Place replacement on top of the bed base and secure side straps under the base and adjust buckles. Ensure that the side straps are not twisted or caught in any of the bed's moving parts
- 3 Position control unit by its hang hooks over foot board of bed. The control unit can also be placed on the floor under the bed.
- 4 Attach inflation hoses to control unit with the press on quick release connectors. Ensure air hoses do not kink between overlay, mattress, bed and control unit.
- 5 Insert power cord into control unit and insert into wall plug.
- 6 Increase pressure setting to maximum and switch on the unit. Activate the static switch and allow up to 20 minutes to inflate completely.
- 7 Once inflated, replace cover sheet loosely from head to foot but can be tucked in under sides of mattress replacement. **SWITCH OFF STATIC FUNCTION**
- 8 Position patient on the system and perform a bottoming out test.
 - a) Choose a deflated cell under patient's sacrum.
 - b) Hand should slip in and out easily along the deflated cell between the patient and the underlying bed base.
 - c) If patient is adequately suspended, then decrease pressure for comfort - allow 3-5 minutes before testing again.
 - d) Refer to incremental pressure adjustment dial and comfort guide on the side of the control unit to assist with the optimum pressure setting.



-  DO NOT USE SYSTEM WITHOUT ITS ZIP ON TOP COVER
-  DO NOT ATTEMPT TO MACHINE WASH OVERLAY
-  KEEP ALL PACKING MATERIAL & INSTRUCTIONS IN THE BAG OR BOX PROVIDED.

See system manual for cleaning instructions.