

EVO CARE CONTACT:

Sales: 07 3355 8000

Fax: 07 3355 5043

Email: sales@evocare.com.au

PO Box 1144, Stafford

Q 4053 Australia

Convuluted Foam (Eg Shell) Overlays

Call or email for more information

More information on our website: <http://www.evocare.com.au/foam-overlays.php>



CONVOLUTED (EGG SHELL) FOAM OVERLAYS

Getting the most from your overlay:

- It is still important to turn your body while using an overlay.
- The overlay works best when placed flat on top of the mattress with convolutions facing up.
- Use a loose fitting light sheet to cover overlay – the overlay is less effective with thick tight bedding over it.
- DO NOT immerse in water as this will effect the performance of the overlay – clean by lightly sponging.

Overlay Specifications		
Description	Product Code	Dimensions (mm)
Single	FQ713144	1900 x 915
Double	FQ000198	1905 x 1375
Queen	FQ000199	2030 x 1524
King Double	FQ710331	2030 x 1830



 **Evocare**[®]

Specialist Equipment Sales & Service

Our catalogue is our comprehensive and up to date website: www.evocare.com.au