
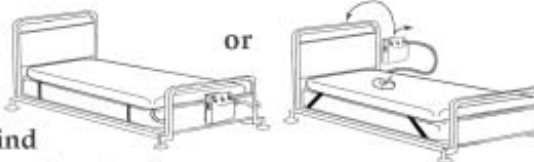
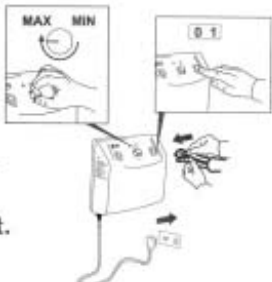
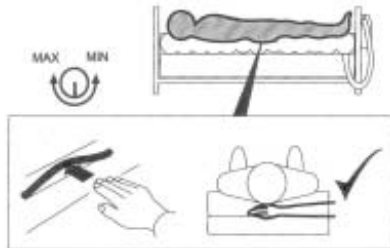







HomeWave II Alternating Overlay Quick Set Up Guide

Note: Allow up to 30 minutes to inflate completely

- 1 Remove all covers and sheets from existing mattress
- 2 Place overlay on top of the existing mattress and secure end straps **COMPLETELY** under the mattress. Ensure that the end straps are not twisted or pulled towards the corners of the mattress.
 
- 3 Position control unit by its hang hooks over foot board of bed. System can also be swapped head to foot and the control unit hung from behind the bed head, mounted on the wall or placed on the floor.
 
- 4 Attach inflation hoses to control unit with the press on quick release connectors. Ensure air hoses do not kink between overlay, mattress, bed and control unit.
 
- 5 Insert power cord into control unit and insert into wall plug.
- 6 Increase pressure setting to maximum and switch on the unit. Allow up to 30 minutes to inflate completely.
- 7 Once inflated, replace any cover sheets loosely from head to foot but can be tucked in under sides of base mattress.
- 8 Position patient on the system and perform a bottoming out test.
 - a) Unzip the top cover down one side of overlay.
 - b) Choose a deflated cell under patient's sacrum.
 - c) Hand should slip in and out easily along the deflated cell between the patient and the underlying mattress.
 
 - d) If patient is adequately suspended, then decrease pressure for comfort - allow 3-5 minutes before testing again.

-  DO NOT USE SYSTEM WITHOUT ITS ZIP ON TOP COVER
 -  DO NOT ATTEMPT TO MACHINE WASH OVERLAY BASE
 -  KEEP ALL PACKING MATERIAL & INSTRUCTIONS IN THE BAG PROVIDED.
- See system manual for cleaning instructions.