


KA360A –(KF36001)– W-TUTOR MS GREY ADULT

USER MANUAL

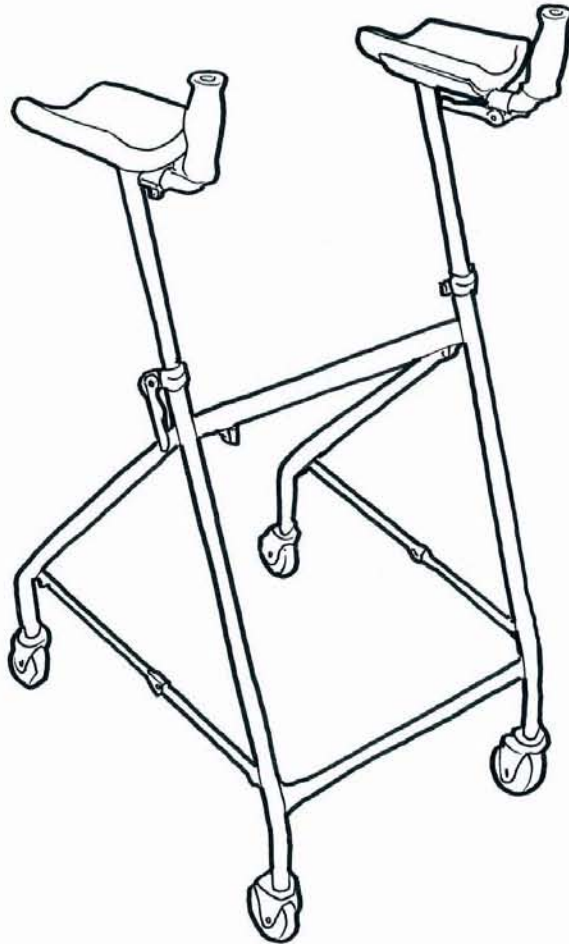

INDEX

1	Specifications
3 – 5	Setup Instructions
6	Folding Instructions
7	Options
8	Maintenance
8	Cleaning
8	Warranty

SPECIFICATIONS


MAXIMUM USER CAPACITY: 100kg

DIMENSIONS:	Height	Minimum	870mm
		Maximum	1270mm
	Width	Front	540mm
		Rear	620mm
	Handgrip to front of forearm support	Minimum	80mm
		Maximum	300mm
	Distance between handgrips	Minimum	430mm
		or Maximum	525mm

WEIGHT: 11kg


CASTORS: 100mm diameter polyurethane wheels

FINISH: Epoxy powdercoated framework
Stainless steel forearm supports
Polyurethane forearm padding
Rubber handgrips

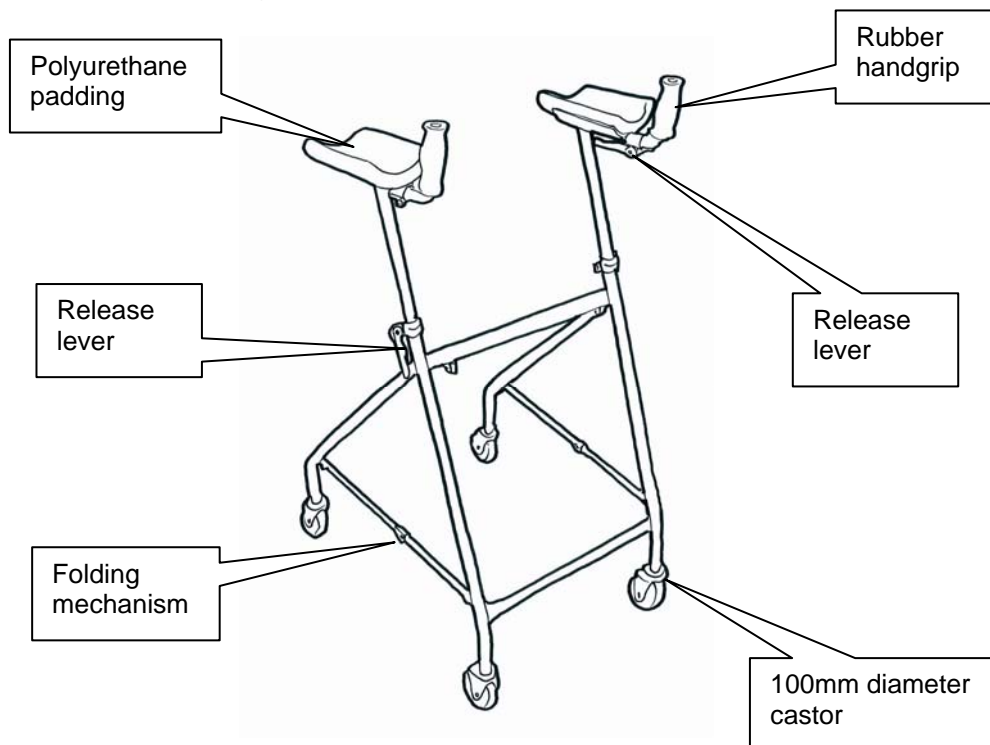


SETUP INSTRUCTIONS

1. Ensuring that the release lever is loose, insert the forearm supports into the main frame and adjusted to the height required by the user.

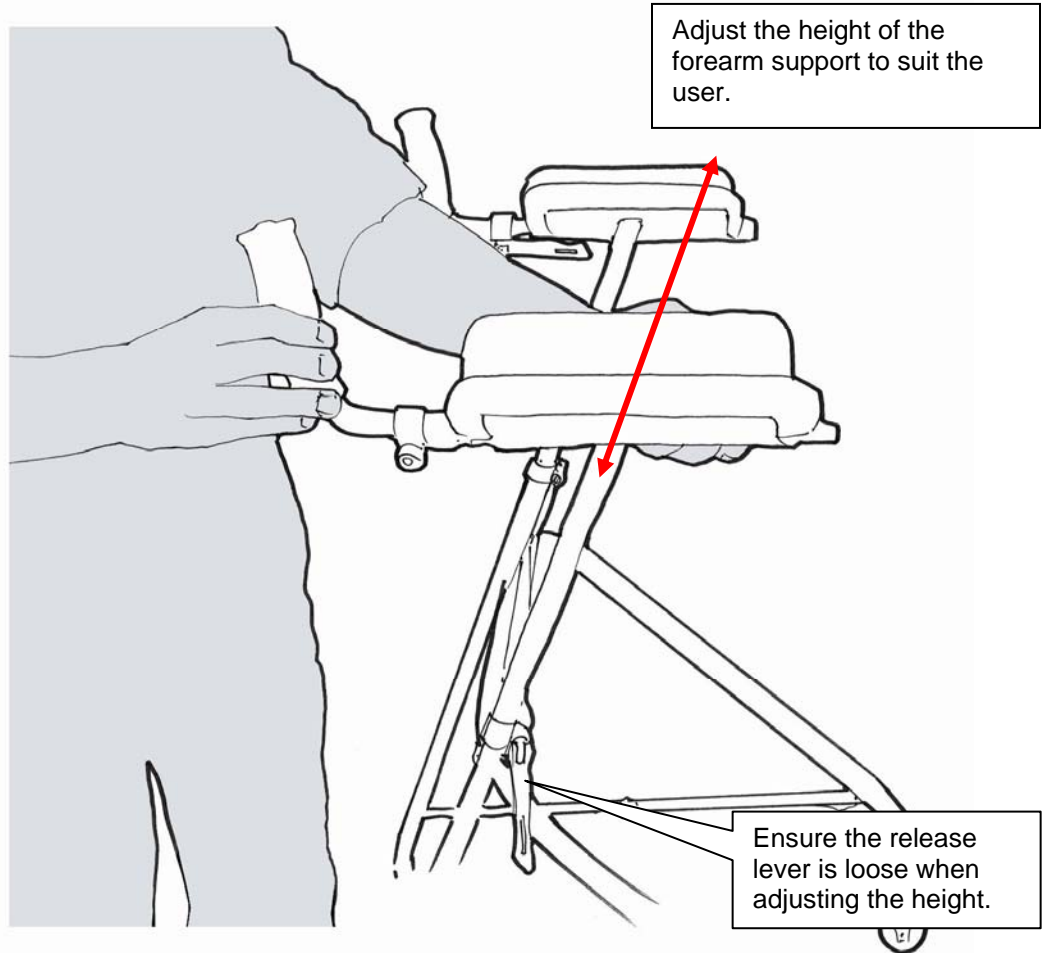


SETUP INSTRUCTIONS Cont.



As the forearm supports are off-set, they can be inserted into the frame to accommodate a user who prefers a wider or narrower frame.

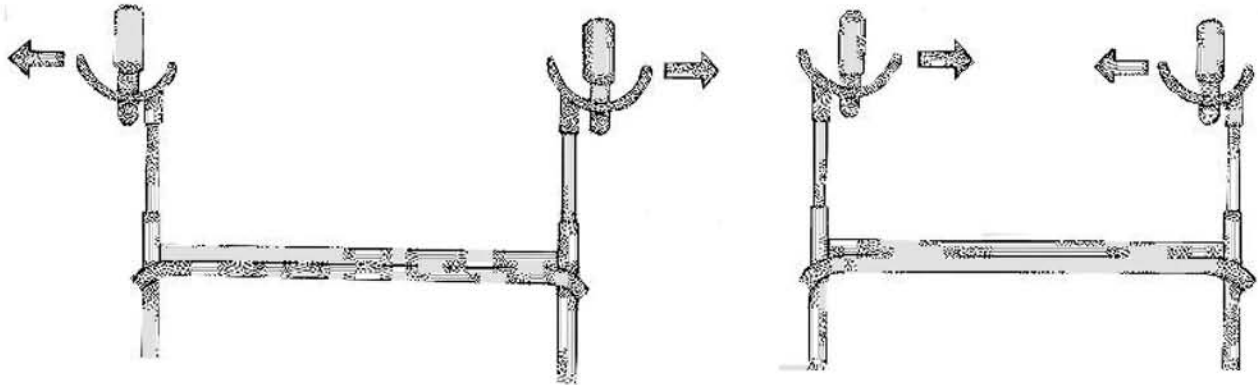


The forearm supports can be used at an angle to the frame for a more comfortable position. Ensure forearm supports are locked in position before use. The handgrip position can also be altered to tilt for a more comfortable position. Ensure handgrip is locked in position before use.

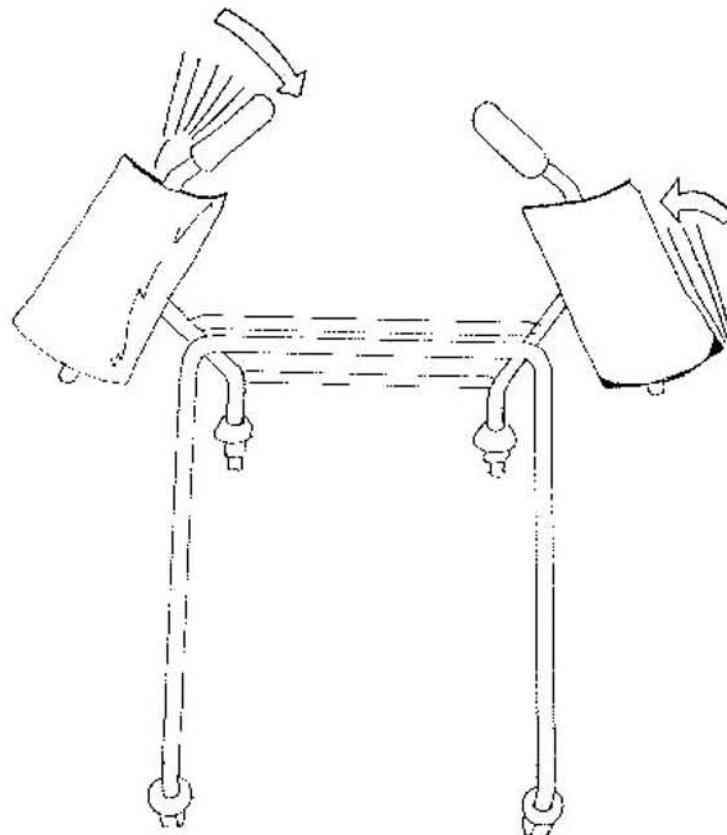

2. Tighten the release lever until it requires some force to lock into position. Test the clamping strength by trying to turn the forearm support. If it moves with little effort, then tighten the release lever more.

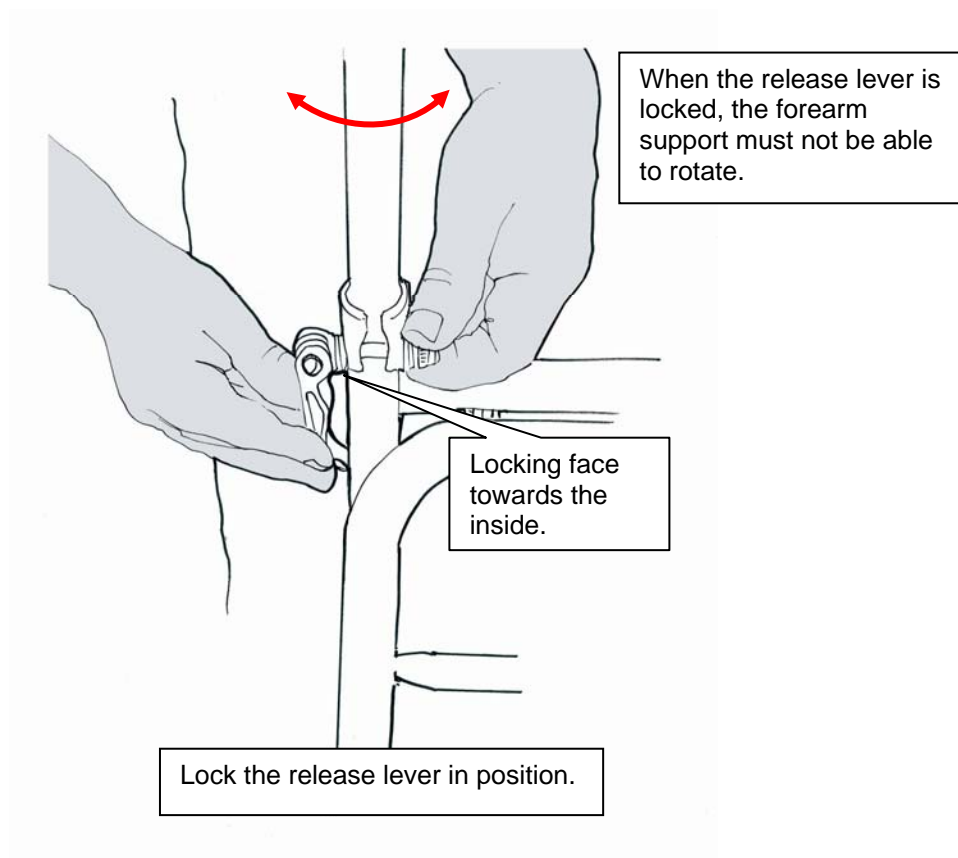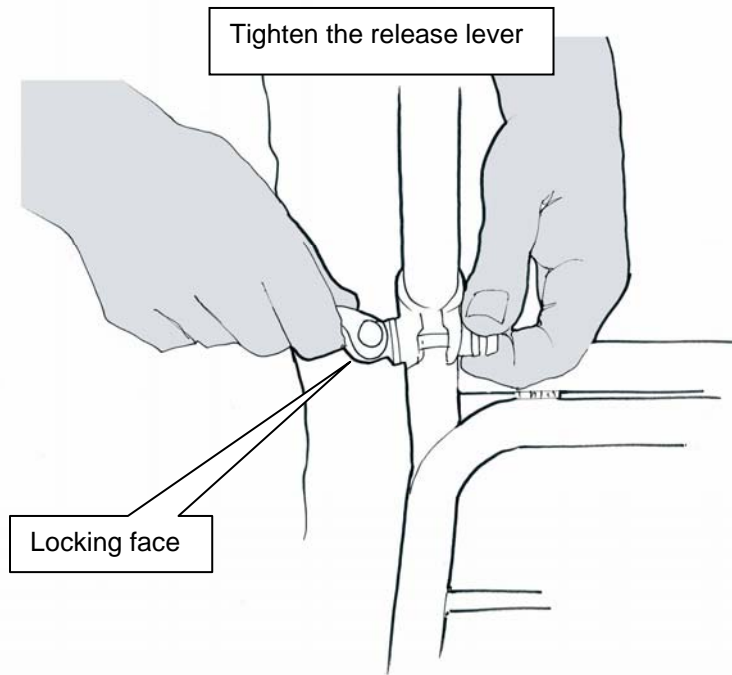


3. Adjust the position of the handgrip relative to the forearm support. Use the same method for adjusting the release lever as described in step 2.

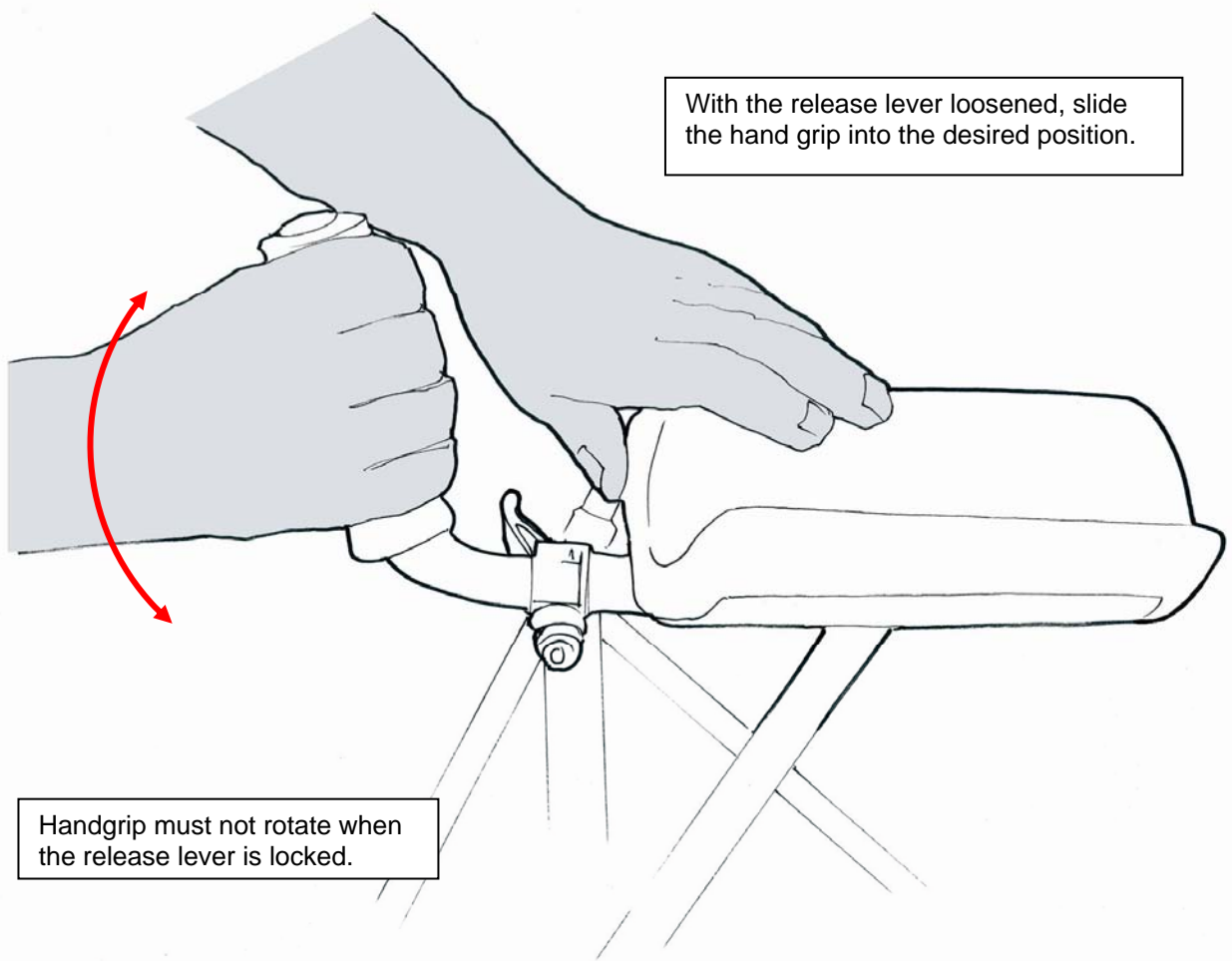


FOLDING INSTRUCTIONS

1. Lift to release the folding mechanism.

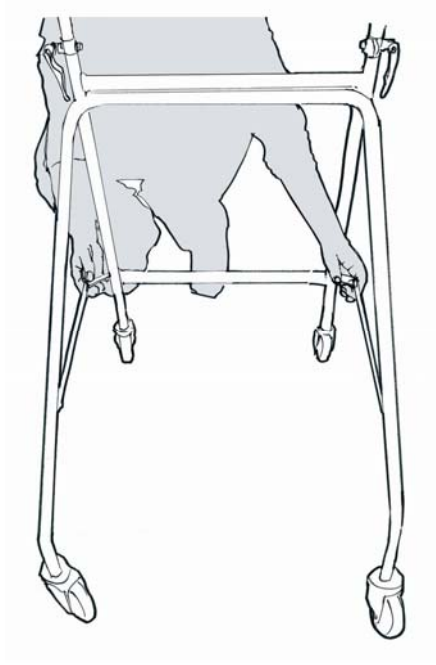

2. Grasp the Forearm supports and lift the frame to completely fold the tutor.

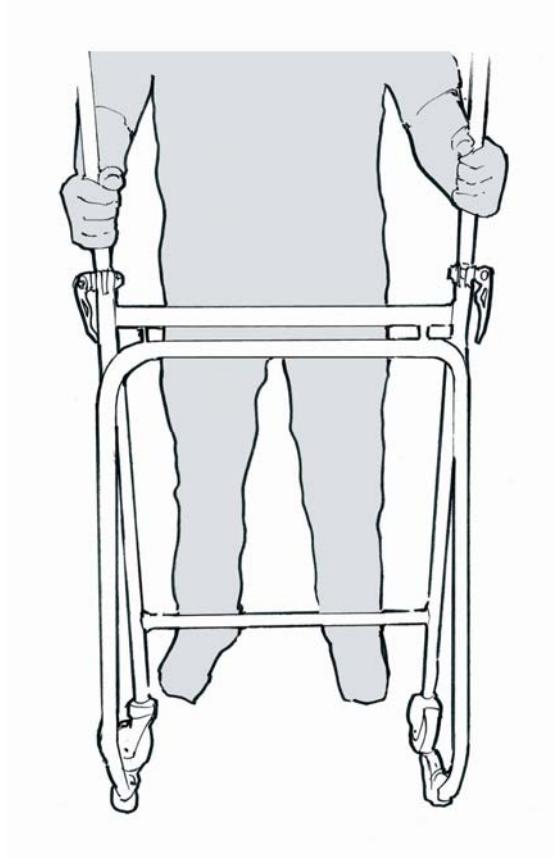

OPTIONS


KS36003 – (KA36APR) - W-TUTOR PAD A-REST UPH ADULT



DIMENSIONS:	Height	Minimum	950mm
		Maximum	1250mm
	Padded rest width		610mm
	Padded rest depth		400mm
	Aperture width		360mm
	Aperture depth		140mm
	Handgrip to front of padded rest		
		Minimum	110mm
		Maximum	400mm
	Distance between handgrips		480mm

KS36006 – (KA36GL) - W-TUTOR R-LEGS GLIDES

Remove rear castors and rivet rear glides in place.

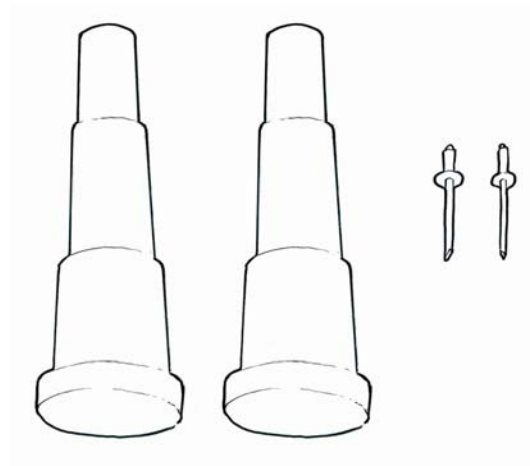

MAINTENANCE

At least monthly, but preferably more frequently.

1. Check that the nuts and screws are tight.
2. Check that the castors are firmly attached to the frame, and freely rotate.

CLEANING

Clean with warm soapy water and soft cloth.

WARRANTY

Hills Healthcare Equipment provides a twelve month structural warranty for this walking tutor.

Hills Healthcare Equipment or an authorized Hills Healthcare Equipment dealer must inspect the product within the warranty period for any claims lodged.

Hills Healthcare Equipment is not liable for any costs borne from shipping of non-conforming product to place of inspection (e.g. distributor address). Hills Healthcare Equipment warranty does not cover product damage accidentally through misuse or neglect.

Hills Healthcare does not warrant product that has been modified from its original form. Due to logistic reasons, it is imperative that you contact your place of purchase who will lodge an official claim on your behalf.

EVOCARE AUSTRALIA PTY LIMITED

A.B.N. 98 078 566 604

Trading as **EVOCARE** and **L&M EQUIPMENT**

P.O. Box 1144, Stafford Qld. 4053

Ph: 07 3355 8000 Fax: 07 3355 5043

Website: <http://www.evocare.com.au>

E-Mail: sales@evocare.com.au

workshop@evocare.com.au



Quality
ISO 9001