

# OWNER'S MANUAL



## BATH BOARD



### LM8330

Thank you for purchasing this Evocare Bath Board. It is strongly recommended that you read through this user manual prior to using your Bath Board for the first time. Your Bath Board should provide years of trouble-free use, provided that you adhere to these recommendations.

**WE ARE HERE TO HELP YOU**

**To request product service and order replacement parts, please call:**

**Customer Service: 07 3355 8000**

**Spare Parts: 07 3355 8007**

**8.30 am to 5.00 pm Monday to Friday**

**Email: sales@[evocare.com.au](mailto:sales@evocare.com.au)**


**If the product does not function properly contact [Evocare](#) Australia.**

When requesting service, please provide the following information:

1. Part Code and Description
2. Date of Purchase

Address: Evocare Australia Pty Limited  
260 South Pine Road, Enoggera, Qld. 4051  
P.O. Box 1144, Stafford, Qld. 4053

## DESCRIPTION

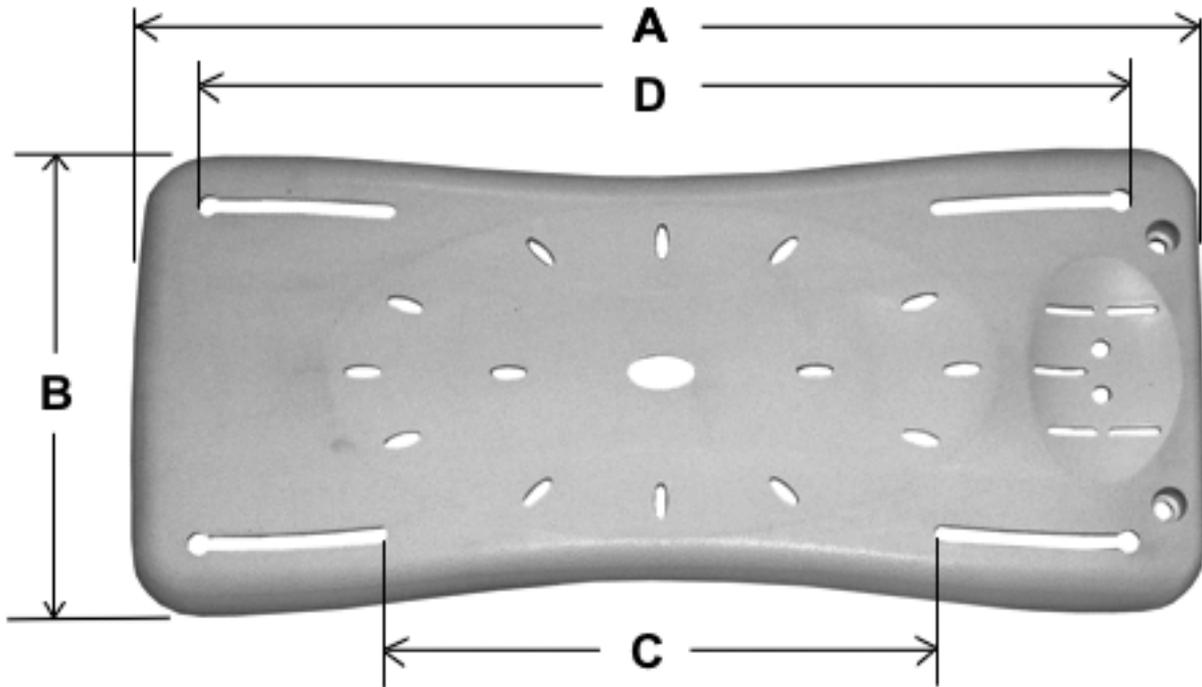

The **LM8330** Bath Board is designed to accept a maximum user weight of **120 kilograms**.

Dimensions:

- |   |       |
|---|-------|
| A. Length of board                        | 700mm |
| B. Width of board                         | 310mm |
| C. Minimum distance between friction feet | 325mm |
| D. Maximum distance between friction feet | 570mm |

Evocare is a quality endorsed company and supplies equipment listed with the Therapeutic Goods Administration.

The ARTG number for the bath Board is [176423](#)

# SAFETY

## SAFETY:

These instructions are guidelines only. Use only as instructed by your healthcare professional.

The bath board is a plastic board that spans the width of the bath and rests on the sides. The bath board is used to enter and exit a bath.

The Bath Board can be slippery when wet—use this product with caution. Product is intended to be used as a sitting surface only. Do not stand on the Bath Board. Do not tip or put an offset load onto the Bath Board or injury may occur. Inspect frequently to ensure all screws are tight. Clean with water and mild soap.


Maximum user weight 120kg

Temperature range: 3 to 70° C

-  **WARNING** Do not install the bath board if there is a shower door track on the side of the bath.
-  **WARNING** Ensure the edge of the bath board against the wall is as close to the wall as possible.
-  **WARNING** The side of the bath against the wall must be wide enough for the bath board to rest safely on it (minimum 25mm to 50mm).
-  **WARNING** It is not recommended using the bath board on free standing baths, or baths with no side wall.
-  **WARNING** When installed the bath board should not exceed the outside wall of the bath by more than 20mm.
-  **WARNING** Supervision or assistance is recommended for users with limited mobility when using the bath board.
-  **WARNING** Ensure the bath board is securely in place before use.
-  **WARNING** Using a slip proof bath mat increases safety, when using the bath board.


## INSTALLATION

1. Secure handle (4) to bath board (1) using handle screws (5) as shown in Figure 1a.

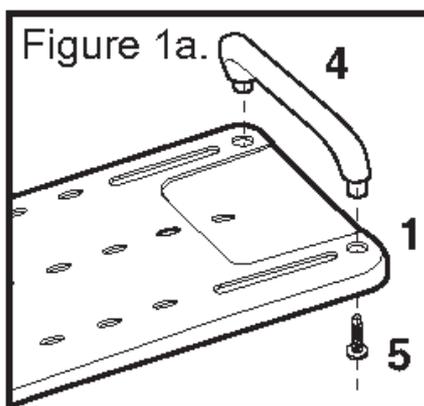

2. Ensure there is a minimum horizontal flat surface of 38mm (X) on the tub rim for the Bath Board overhang to rest upon as shown below. Adjust friction feet to fit snugly inside bathtub (a). Tighten the screw to secure friction feet in place (b).

**Note: The side of the Bath Board with the handle MUST be closest to the wall.**

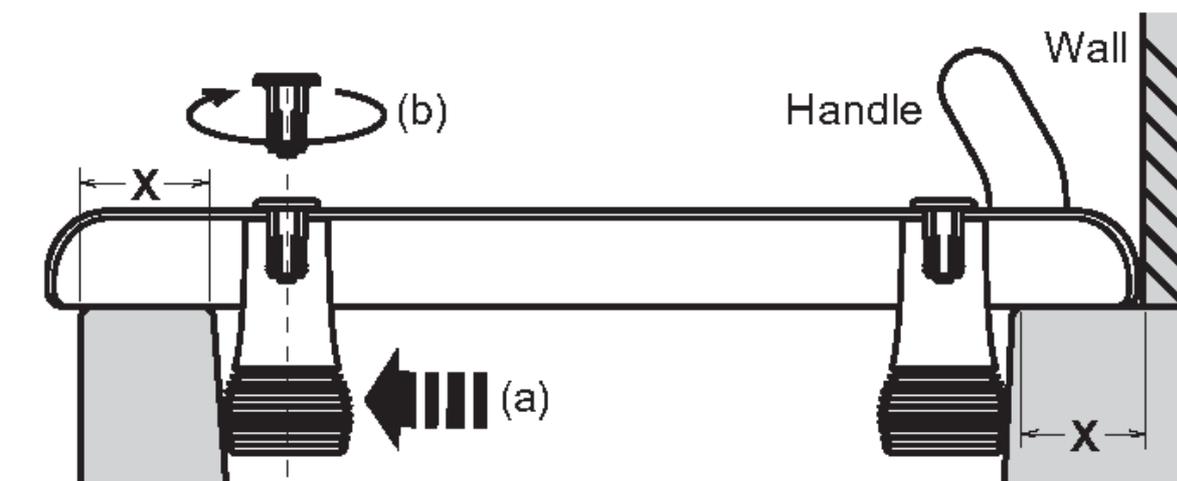


# PARTS DIAGRAM

## PARTS DIAGRAM

1. Bath Board
2. Friction Feet (x 4)
3. Friction Feet Screws (x 4)
4. Handle
5. Handle Screws (x 2)
6. Anti-Slip Tabs (x 2)

**Item 2 and 3 Combined Code:**  
**LM8330AC** One Friction Foot  
& Screw

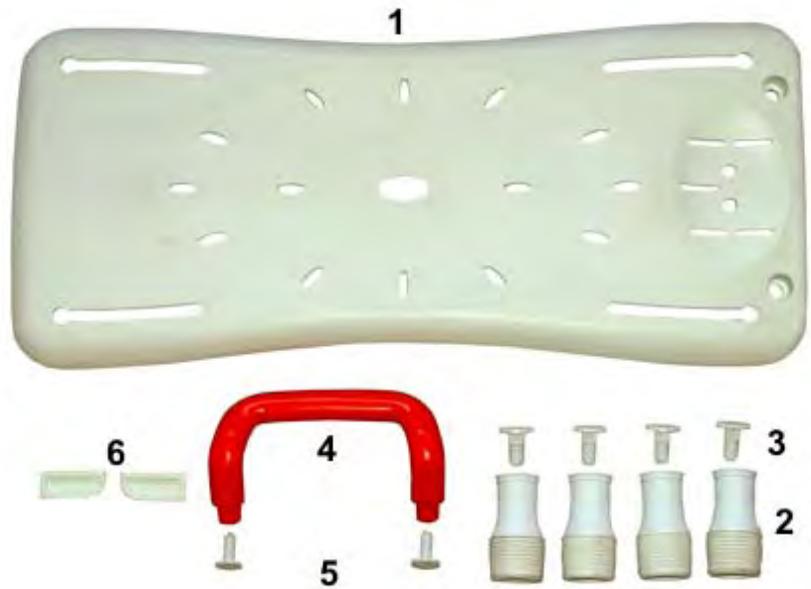

1. Evocare provides consumers with the following warranty in relation to this Product, in addition to complying with the requirements of any relevant legislation, including the *Competition and Consumer Act 2010* (Cth) in Australia and the *Consumer Guarantees Act 1993* in New Zealand (the Acts), except where a New Zealand consumer acquires the Product for the purposes of a business.
2. In this warranty, we have used the following definitions:
  - (a) **Form** means a warranty claim form issued by Evocare in respect of Products;
  - (b) **Evocare** or **our** means Evocare Australia Pty Limited, ABN 98 078 566 604 of Level 19, 144 Edward Street, Brisbane 4000;
  - (c) **Products** means the goods manufactured for Evocare (including products manufactured by its contract manufacturers);
  - (d) **Material** means a material or component used by Evocare in the manufacture of the Products;
  - (e) **Retailer** means the authorised dealer of Products from whom the Product was purchased;
  - (f) **Warranty Period** means the warranty period commencing from the date of purchase of the relevant Product. If the Product or part (as the case may be) is repaired or replaced, there will be no extension to the original warranty period; and
  - (g) **Workmanship** means the handling, assembly and manufacturing processes performed by or on behalf of Evocare in order to manufacture the Products.
3. Evocare warrants that for the duration of the Warranty Period, all Products will be free of faults arising from defects in Workmanship or Materials, on the terms and conditions set out in this warranty.
4. Evocare undertakes that if during the Warranty Period any Product, or any part of a Product, has failed to operate correctly due to faulty Workmanship or defective Material, it will repair or replace the Product or part (as the case may be) free of charge provided that the following procedures are met :
  - (a) The consumer must contact the Retailer or Evocare upon becoming aware of any defect to a Product upon which the consumer will be required to complete a Form and provide satisfactory proof of purchase.
  - (b) Evocare will review the Form to determine whether there is a defect, and if so Evocare agrees to (at its option) repair, replace or supply equivalent goods, or pay the cost of any of those remedies to the consumer.
  - (c) If Evocare requests the return of the applicable Product or part, the consumer will be responsible for the collection and freight costs of returning that Product or part to Evocare.
  - (d) The consumer acknowledges that it is also responsible for the freight costs to deliver any new Product or replacement part to it.
5. This warranty is in addition to any non-excludable legal rights or remedies conferred on the consumer under any applicable Act and any similar laws. To the extent permitted by law, Evocare's liability for any non-excludable condition or warranty is limited to rectifying any defect at its option, as set out in paragraph 4(b).
6. Subject to the requirements of any applicable Act or legislation and to the extent permitted by law, no liability (whether expressed or implied) of any nature whatsoever, is accepted by Evocare for any consequential loss, damage or injury arising as a result of any fault in the Products.
7. This warranty does not extend to damage to Products which occurs during transit or transportation, or which is caused by any abuse, accident or improper installation, connection, use, adjustment or repair or use of goods otherwise than in accordance with instructions issued by Evocare.
8. The warranty on Products is waived if any addition or attachment to the Products do not have Evocare's approval or are not sold as Evocare products. The Products are designed to perform specific tasks under established test loads and unauthorised attachments may produce stresses for which the design is not appropriate.
9. The following applies to consumers who purchased a relevant Product in Australia:

*Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement or refund for a major failure and compensation for any other reasonably foreseeable loss or damage. You are also entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and the failure does not amount to a major failure.*

Evocare warrants this Bath Board for a period of twelve months.

**NOTE:** This does not include staining or accidental damage from misuse or neglect.

Evocare must inspect the product within the warranty period for any claims lodged.

Evocare warranty does not cover product that has been modified from its original form.

For more information about your LM8330 Bath Board, warranty or to order parts call 07 3355 8007.

E- mail: [sales@evocare.com.au](mailto:sales@evocare.com.au)  
Website [www.evocare.com.au](http://www.evocare.com.au)

**Manufactured for:**

Evocare Australia Pty Limited, 260 South Pine Road, Enoggera. QLD. 4051.

Phone: 07 3355 8000

FAX: 07 3355 5043