



# WASHER DISINFECTOR



[www.evocare.com.au](http://www.evocare.com.au)

Phone: 07 33558000 Fax: 07 33555043



## ENSURE YOUR UTENSILS ARE READY FOR HANDLING & USE WITH AUTOMATIC CLEANING AND THERMAL DISINFECTION

### SAFETY

A computer monitored door lock mechanism, ensures it cannot be opened until the cycle is complete.

### USER FRIENDLY

Single button cycle start.

### RELIABILITY

Equipped with an "intelligent" microprocessor developed for Malmet, it incorporates the fine-tuning of experience with the best features of heavy-duty industrial electronics.



**WDT2**

### COMPLIANCE

Meets Australian Standards AS2945  
Washer Disinfectors for Hospital Utensils.  
Watermark WMKT21156

### TOP LOADING CAPACITY

Designed to hold 6 large wash basins or a varied load of smaller items, eg: kidney dishes, sputum bowls, tooth mugs etc. (Special instrument baskets are an optional extra).

### THE CYCLE OF OPERATION

1-	Cold Water Wash to remove soluble organic materials.
2-	Hot Water Wash with detergent added automatically.
3-	Hot Water Rinse.
4-	Hot Water Disinfection.
5-	Cold Water Rinse.



 **Evocare**<sup>®</sup>

[www.evocare.com.au](http://www.evocare.com.au)

Phone: 07 33558000 Fax: 07 33555043

\* ACCESSORIES AND PRODUCTS SHOWN IN ALL PHOTOS ARE NOT INCLUDED WITH UNITS.

# PRODUCT SPECIFICATIONS

Model		WDT2 - Top Loading
Electricity Supply - (Australia Only)		240V 18Amp 50Hz
Water Supply		Cold and Hot (min 43°C)
Dimensions (mm)		655W x 840D x 1030H
Internal Dimensions		595W x 480D x 595H
Water Capacity		18.5 litres
Pump Circulation	Electrical rating	0.55 kW
	Capacity at head	2.25m <sup>3</sup> /hr at 31m head
Pump drain	Electrical rating	0.16 kW
	Capacity at head	45 litres/min @ 1.25m head
Pump detergent	Type and make	Ub 260 C
Detergent usage		Approx. 65ml per cycle
Detergent type		Malmet or equivalent
Temp display		0 - 100°C coil meter
Thermistor probe		Resistance 22k ohms at 25°C
Electric element rating		3.6kW 240V
Cold water	Consumption /cycle	32 litres
Hot water	Consumption /cycle	48 litres
PCB		Yes
Electrical rating of unit		240 V 18A full load
Cycle times		Approx. 28 minutes with 60°C incoming hot water at 350 kPa
Disinfecting times		70°C - 15 minutes
		75°C - 10 minutes
		80°C - 5 minutes
		85°C - 2 minutes
Materials	Wash chamber & external panels	304 stainless steel
	Base tray & frame	304 stainless steel

# MALMET (Australia)

PTY LTD

ABN 95 001 717 791



Malmart (Australia) Pty Ltd is a leading manufacturer of specialised hospital equipment for CSSD, Recovery, Obstetrics, Theatre and Ward Utility Rooms. Malmart is a privately owned Australian family business who prides itself as producing high quality products.

Malmart has been manufacturing specialised hospital equipment since 1969 and supplies hospitals and nursing homes throughout Australia and overseas.

Extensive and continual research and development ensures Malmart products remain the most reliable and user friendly equipment available. Malmart products are manufactured at our Leeton Site which is in the "Heart of the MIA" in New South Wales. The Leeton Manufacturing Site is ISO 9001 certified, this gives our customers peace of mind that they are getting quality products each and every time.

HEAD OFFICE & CUSTOMER SERVICE  
PO Box 373 Leeton NSW 2705 Australia



AUSTRALIAN MADE  
AUSTRALIAN OWNED



 **Evocare**<sup>®</sup>  
www.evocare.com.au

Phone: 07 33558000 Fax: 07 33555043

**WARNING:**

*This brochure contains general information only. If you require certified and accurate dimensions or any other details which may affect your installation, please contact Malmart. E.&O.E.*

*In the interest of continued product improvement, Malmart reserves the right to alter design and product specifications without notice. E.&O.E. For further information on warranty terms and conditions, please refer to Malmart's Warranty Statement.*