

Quad-base Sticks

Designed for walking stick users requiring a higher level of support. Constructed of strong, lightweight aluminium. Supplied with anti-slip rubber tips and with swan neck handles.

LM3843 Black Quad Stick small low base — Swan Neck handle.
Adjustable 800—1003mm SWL 100kgs
Dimensions of base footprint — 15cm x 10cm (6" x 4")
ARTG No: 200852

The **Swan Neck** grip handle brings the handle over the top of the shaft of the walking stick allowing the user to put more weight onto the walking stick. As the pressure is straight down, the rubber ferrule is less likely to slip. The swan neck grip handle is the best handle for the unstable user who puts a lot of weight on their walking stick.

Using a Quad Stick

- A Quad Stick provides good stability.
- They can also be used on stairs, unlike walkers.
- A Quad Stick can stand next to you and will not fall over when you sit or need both hands.
- Do not use the Quad Stick to pull yourself out of the chair as it may tip over and cause injury.
- Stand up first using the chair, and then use the Quad Stick handle.
- It is important to discuss with a therapist before deciding whether a small base or large base is best.



LM3843