

RippleWave Bubble Mat

Quick Set Up Guide

RYNIKAN PTY LTD A.B.N. 98 078 566 604

Trading as **L&M EQUIPMENT**

Sales & Accounts (07) 3355 8000

Workshop & Warehouse (07) 3355 8004

Fax (all departments) (07) 3355 5043

E-mail sales@landm.com.au

workshop@landm.com.au

warehouse@landm.com.au

accounts@landm.com.au




RippleWave Bubble Mat Quick Set Up Guide

Note: Allow approx. 20 minutes to inflate completely


1 Remove all covers and sheets from existing mattress

2 Place bubble mat on top of the existing mattress and secure end flaps COMPLETELY under the mattress.

3 **IMPORTANT!**
Do not tuck cell sections under ends of mattress but allow excess length to lie over mattress surface. Overlay will compress to a shorter length once inflated

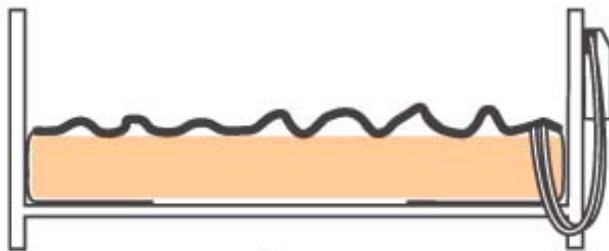

Side view - prior to inflation

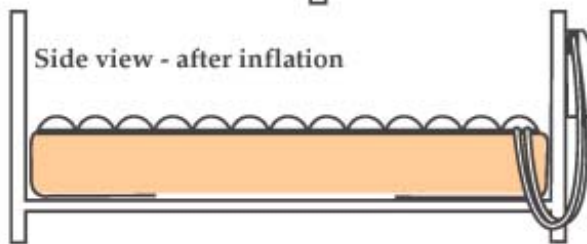

Side view - after inflation

4 Position control unit by its hang hooks over foot board of bed. System can also be swapped head to foot and the control unit hung from behind the bed head, mounted on the wall or placed on the floor.

5 Attach inflation hoses to control unit with the press on connectors. Ensure air hoses do not kink between overlay, mattress, bed and control unit.

6 Insert power cord into control unit and insert into wall plug.

7 Increase pressure setting to maximum and switch on the unit. Allow up to 30 minutes to inflate completely.

8 Once inflated, replace any cover sheets loosely from head to foot but can be tucked in under sides of base mattress. Adjust pressure settings for comfort.

