



IN AN EMERGENCY, PRESS THE EMERGENCY CYCLE STOP BUTTON


ONCE THE CYCLE HAS STARTED, IT CANNOT BE INTERRUPTED

BE AWARE THAT THIS WILL CAUSE THE DEVICE TO GO TO FAULT AND A
SERVICE TECHNICIAN WILL BE REQUIRED TO RESET DEVICE



DO NOT PRESS POWER OFF DURING CYCLES


TO SELECT RELEVANT CYCLE

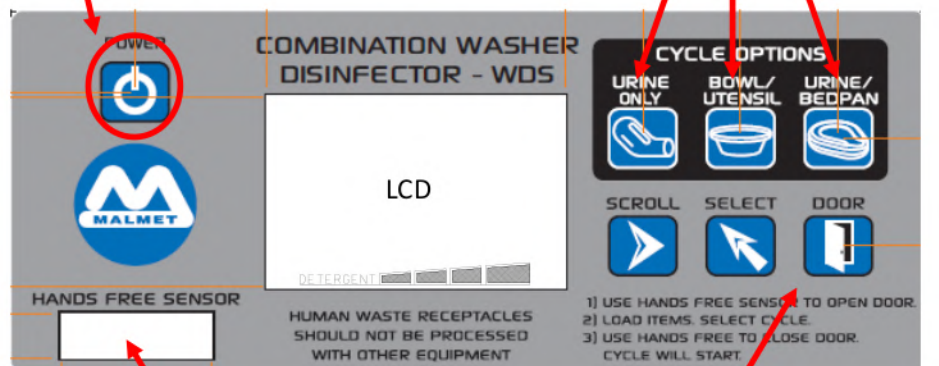

HAND FREE SENSOR
TO OPEN OR CLOSE DOOR

MANUALLY OPEN & CLOSE DOOR

STEP 1 OPEN DOOR

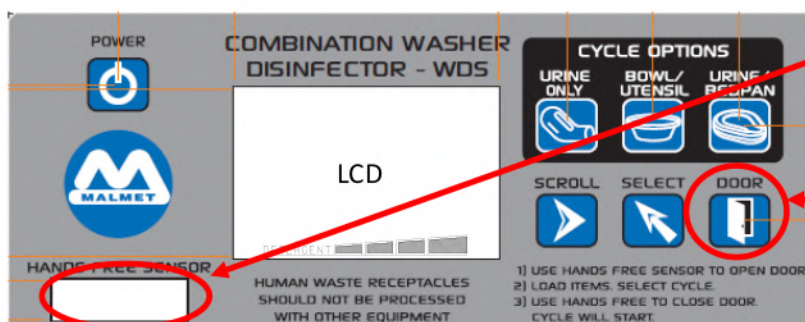

HOLD HAND OVER
'HANDS FREE SENSOR'
Or
'DOOR' BUTTON



WDS INSTRUCTIONS

FOR URINAL BOTTLES ONLY



STEP 2 SET RACKS AS REQUIRED



**BEDPAN RACK MUST BE IN THE
DOWN POSITION**



**BOWL RACK MUST BE UP AGAINST
BACK WALL OF CHAMBER**

STEP 3 LOAD URINAL BOTTLES



STEP 4 SELECT & START CYCLE



**PRESS THE RELEVANT CYCLE
BUTTON TO START CYCLE.**

HANDS FREE SENSOR



**ACTIVATE SENSOR TO CLOSE DOOR AND
AGAIN WHILST "START" IS FLASHING TO
START CYCLE**



WDS INSTRUCTIONS

FOR BOWLS AND/OR UTENSILS



STEP 2 SET RACKS AS REQUIRED



**BEDPAN RACK MUST BE IN THE
DOWN POSITION**




**BOWL RACK MUST BE IN
DOWN POSITION**

STEP 3 LOAD BOWLS AND/OR UTENSILS



**KIDNEY DISHES ON
KIDNEY DISH RACK**

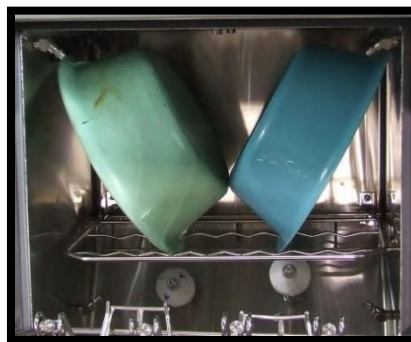

LARGE BOWLS



MEDIUM BOWLS

STEP 4 SELECT & START CYCLE



**PRESS THE RELEVANT CYCLE
BUTTON TO START CYCLE.**

HANDS FREE SENSOR



**ACTIVATE SENSOR TO CLOSE DOOR AND
AGAIN WHILST "START" IS FLASHING TO
START CYCLE**



WDS INSTRUCTIONS

FOR URINAL BOTTLES AND/OR BEDPANS



STEP 2 SET RACKS AS REQUIRED




BEDPAN RACK UP & LOCKED IN POSITION




BOWL RACK MUST BE UP AGAINST BACK WALL OF CHAMBER

STEP 3 LOAD URINAL BOTTLES AND/OR BEDPANS

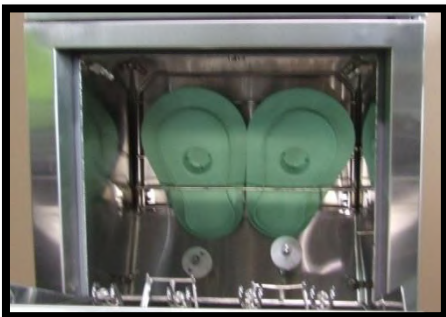

BEDPAN LIDS INTO BACK OF BOWL RACK

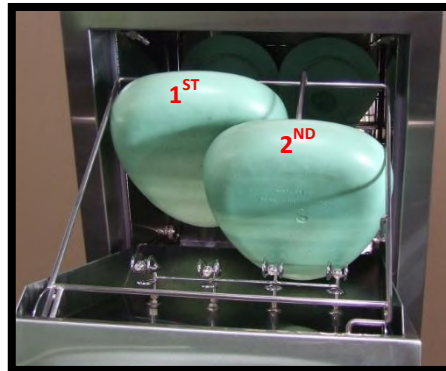

BEDPAN OR SLIPPER PANS ONTO BOTTOM HOLDER OF RACK



URINAL BOTTLES IN RACK

STEP 4 SELECT & START CYCLE



PRESS THE RELEVANT CYCLE BUTTON TO START CYCLE.

HANDS FREE SENSOR



ACTIVATE SENSOR TO CLOSE DOOR AND AGAIN WHILST "START" IS FLASHING TO START CYCLE