




MALMET ES-D BEDPAN WASHER DISINFECTOR

Quick Reference Guide

For Complete Instructions please refer to the
Operation, Maintenance and Installation Manual for this unit.

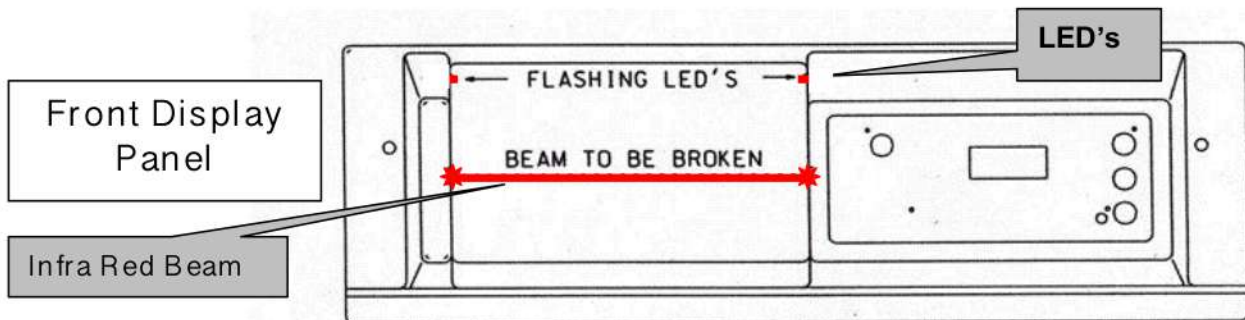


OPERATING MACHINE

1. **OPEN** DOOR – BREAK BEAM WITH **CLOSED FIST** or PAN/BOTTLE
2. **LOAD** PAN & BOTTLES AS PER INSTRUCTIONS (Page 9)
3. **CLOSE DOOR & START CYCLE** - BREAK BEAM WITH **CLOSED FIST**
 - a. DOOR WILL CLOSE Machine displays **rdY**
 - b. **BREAK BEAM** within 8 seconds with **CLOSED FIST** and cycle will start.

If not broken in 8 seconds beam must be broken **TWICE** to reopen.

BEDPAN INSERTION

Refer Page 9 in the Operation, Maintenance and Installation Manual.

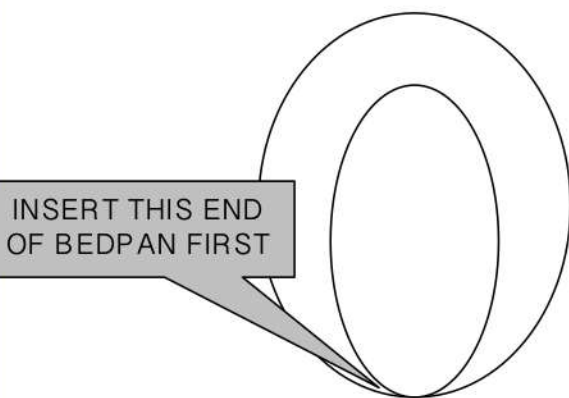

STANDARD SLIPPER PAN INSERTION

Pan must be firmly pushed down and locked into rack.

Refer Page 9 in the Operation, Maintenance and Installation Manual.

