



Allura II



Australian Distributor

EVOCARE AUSTRALIA PTY LIMITED

A.B.N. 98 078 566 604

Trading as **EVOCARE** and **L&M EQUIPMENT**

P.O. Box 1144, Stafford Qld. 4053

Ph: 07 3355 8000 Fax: 07 3355 5043

Website: <http://www.evocare.com.au>

Email: sales@evocare.com.au

workshop@evocare.com.au

warehouse@evocare.com.au

accounts@evocare.com.au



Burmeier GmbH & Co. KG

(Ein Unternehmen der Stieglmeyer-Gruppe)


Pivitsheider Straße 270 • D - 32791 Lage/Lippe

Tel. : 0 52 32/98 41- 0 • Fax: 0 52 32/98 41- 41

Email: service-zentrum@burmeier.de




Stand: 01.01.2015

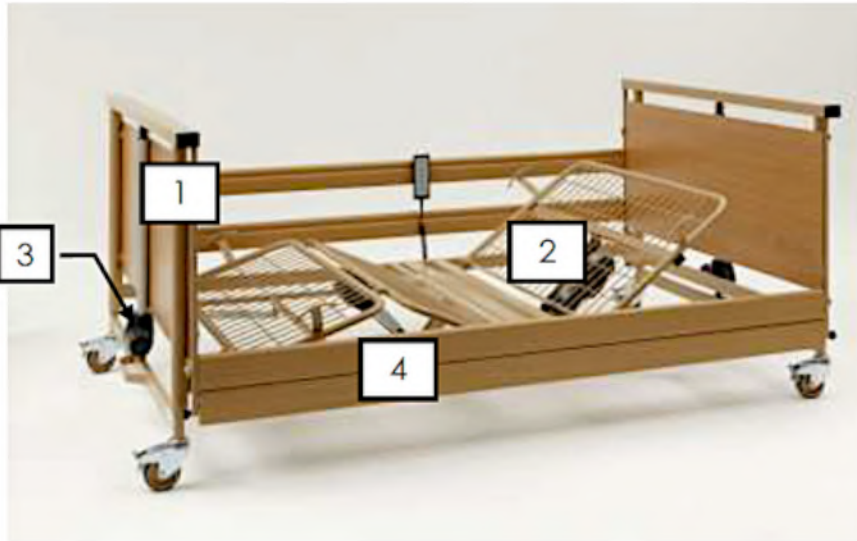



Allura II Care Bed (parts for 100 & 120 unless specified)

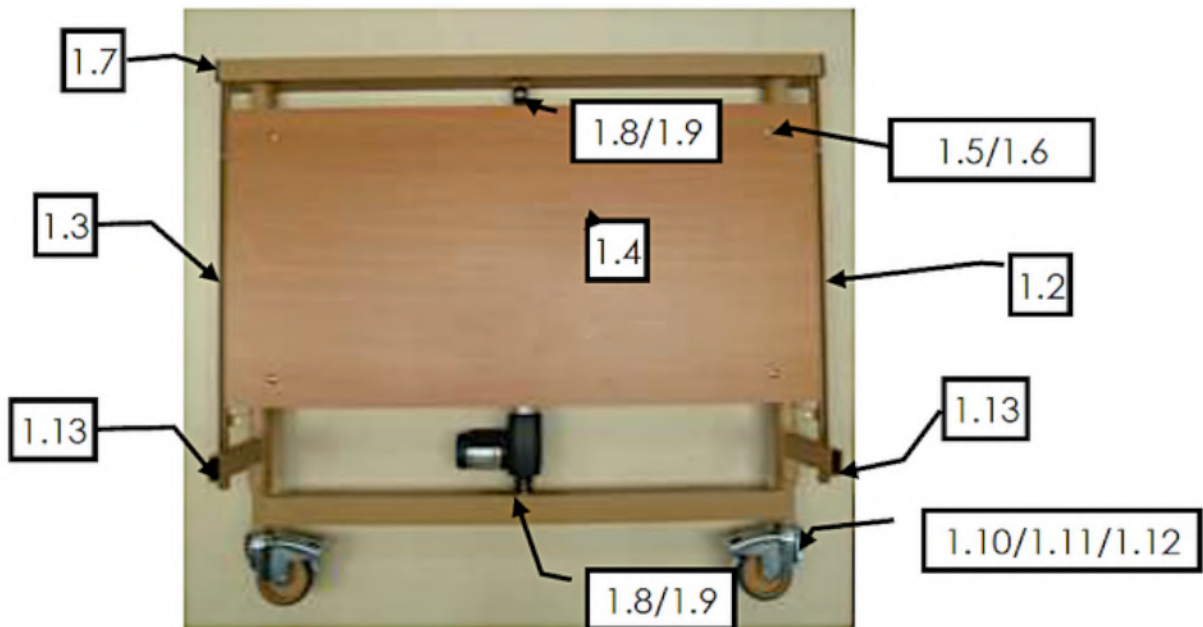1.1 Chassis ends



1.1 Chassis ends

1.1		chassis complete with wood end and motor (model 100)	238438
1.1		chassis complete with wood end and motor (model 120)	238439

1.1 Chassis ends


1.2		Right side rail guide	214893
1.3		Left side rail guide	214894
1.4		Filling Panel 1060 x 530 x 12mm (model 100)	190875
1.4		Filling Panel 1260 x 530 x 12mm (model 120)	188457
1.5		Sleeved nut 13 x 8 x 10 x M6	223671
1.6		Bolt M6 x 12	166107
1.7		Plug 60 x 40 black, for cross bar	188415
1.8		Bolt for lifting motor Ø10 x 35mm	182476

1.1 Chassis ends

1.9		Locking washer for lifting motor Ø10mm	102555
1.10		castor Ø100mm	167036
1.11		Bolt M10 x 30mm	101014
1.12		Serrated lock washer Ø 10.5mm	209395
1.13		Tri-Grip bolt M8 x 18mm	152251

2 mattress base			
2		Metal slat mattress base (200 x 100cm) with / without motor (model 100)	230146
2		Metal slat mattress base with motors (model 120)	230148

2.1 mattress base head section

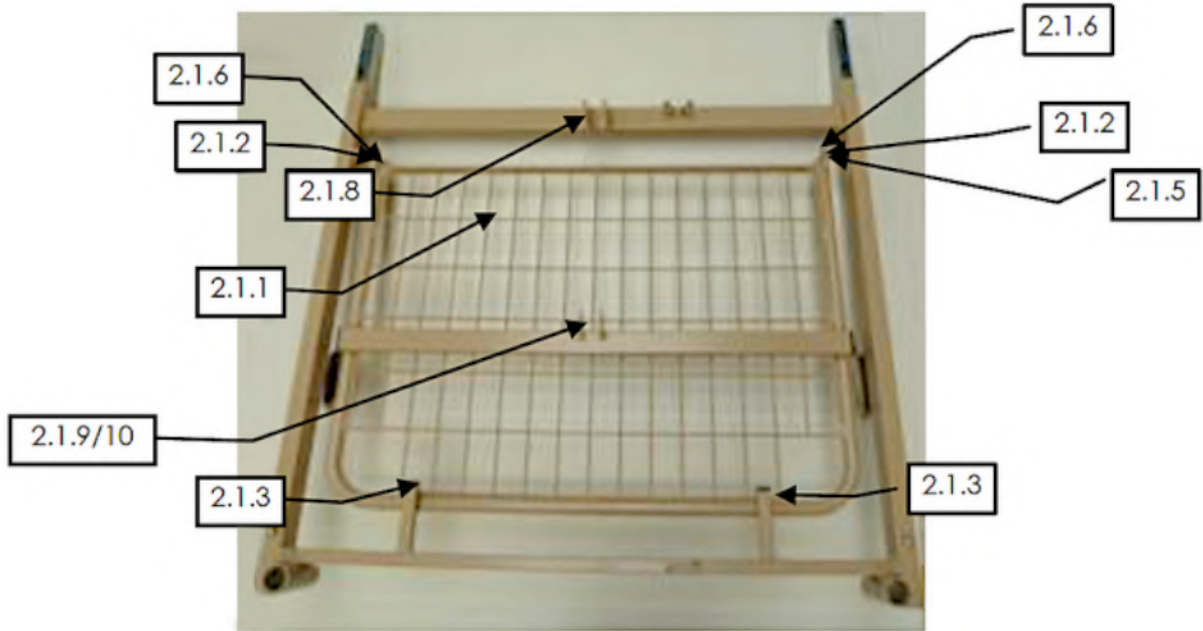


2.1 mattress base head section			
2.1		Mattress base head section without motor (model 100)	242685
2.1		Mattress base head section without motor (model 120)	241978
2.1.1		Backrest 780 x 900mm (model 100)	238138


2.1 mattress base head section

2.1.1		Backrest (model 120)	238139
2.1.2		Plug, round Ø25mm	181484
2.1.3		Black square plug, plastic 20 x 20mm	100105
2.1.4		self-adhesive buffer	106738
2.1.5		rounded head screw (allen) M8 x 20mm	205651
2.1.6		collar bush	194804
2.1.7		black square plug, plastic 40 x 20 x 3mm	185239

2.1 mattress base head section



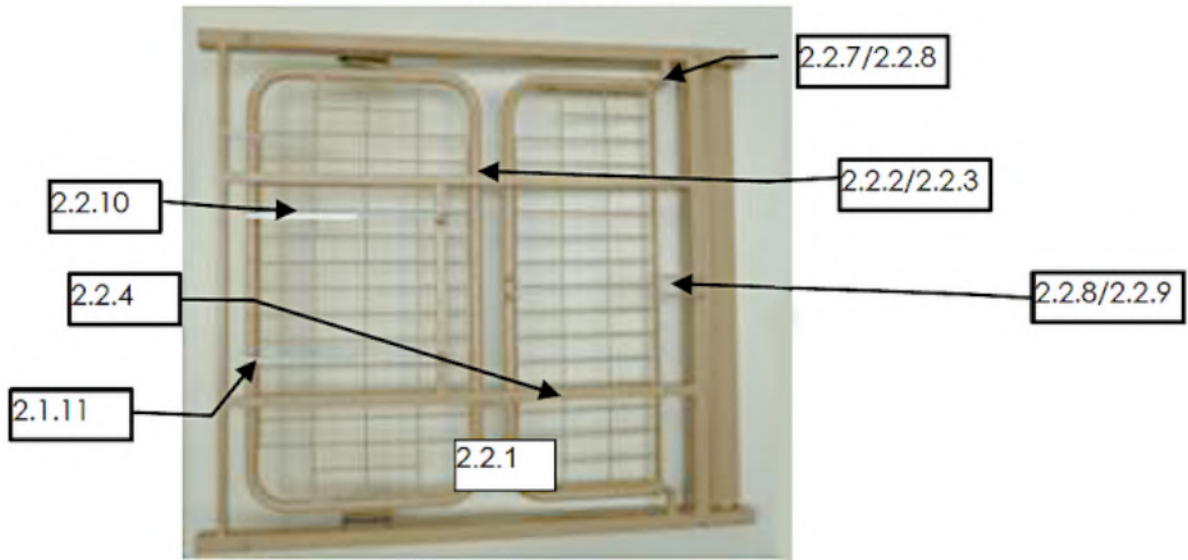

2.1.8		quick release bolt Ø10 x 60mm	200395
2.1.9		collar stud Ø10 x 50mm	171801
2.1.10		lock washer (open) Ø10mm	102555

2.2 mattress base foot section



2.2 mattress base foot section			
2.2		Mattress base foot section without motor (model 100)	242687
2.2		Mattress base foot section without motor (model 120)	241979
2.2.1		Leg section (model 100)	238141
2.2.1		Leg section (model 120)	238142
2.2.2		blind rivet Ø6.4 x 16.5mm, (connection thigh / lower leg rest)	179334

2.2 mattress base foot section

2.2.3		washer, metal Ø6.4mm (connection thigh / lower leg rest)	106043
2.2.4		Self-adhesive white stripe 1000 x 18 x 1mm	192516
2.2.5		Plug, round Ø25mm	181484
2.2.6		rounded head screw (allen) M8 x 20mm	205651
2.2.7		collar bush	194804
2.2.8		collar stud Ø10 x 50mm	171801
2.2.9		lock washer (open) Ø10mm	102555
2.2.10		ratchet	150319
2.2.11		Locking washer Ø8mm	181858

3. Drive System




3.1		hand control with locking function	231675
3.1.1		hook for hand control	158919
3.1.2		locking key	167882
3.2		Switch-Mode SMPS19	209738
3.3		Open bus control including cable	235924
3.3.1		Cable holder	211082

3. Drive System

3.3.2		Control bracket	211761
3.4		Backrest motor Linak	214151
3.5		thigh rest motor Linak	214152
3.6		lifting motor Linak	187371
3.7		Connection cable D400 Lift motor control	187372
3.8		Connection cable D200 Backrest motor control box	159602
3.9		Connection cable D400 thigh rest motor control	187372

4 safety sides			
4.1		safety side guide	188731
4.1.2		Metal clip 330mm for safety side guide	191714
4.2		safety side, timber 2025 x 95.5 x 30mm	188334

5. lifting pole

5. lifting pole			
5		lifting pole	146602
5.1		lifting pole without grab handle	160802
5.2		grab handle	222225
5.3		lifting pole end cap, topas	182111

Accessories

Number	designation	article number
A	lifting pole with grab handle	146602
B	padded side rail cover for 2-bar side rail	225921
C	bed extension approx. 20cm, colour: topaz incl. 4 longer side rails 2225 x 95.5 x 33mm	195915 / 4 x 198167
D	reading lamp with fastening (claw) at lifting pole	175591 / 174231
E	reading lamp with adapter for the lifting pole sleeve	196221
F	storage and transport aid, colour: topas	196424
G	infusion holder (5200-28-50)	116354
H	wall deflector roll including bracket	206751
I	upholstery pad 100 x 200 x 12cm RG 40	242024
J	upholstery pad 100 x 220 x 12cm RG 40	242025
K	stand up aid	244029