




BURMEIER

List of spare parts / accessories Dali Low Entry (24V) as of the year of manufacture 2011

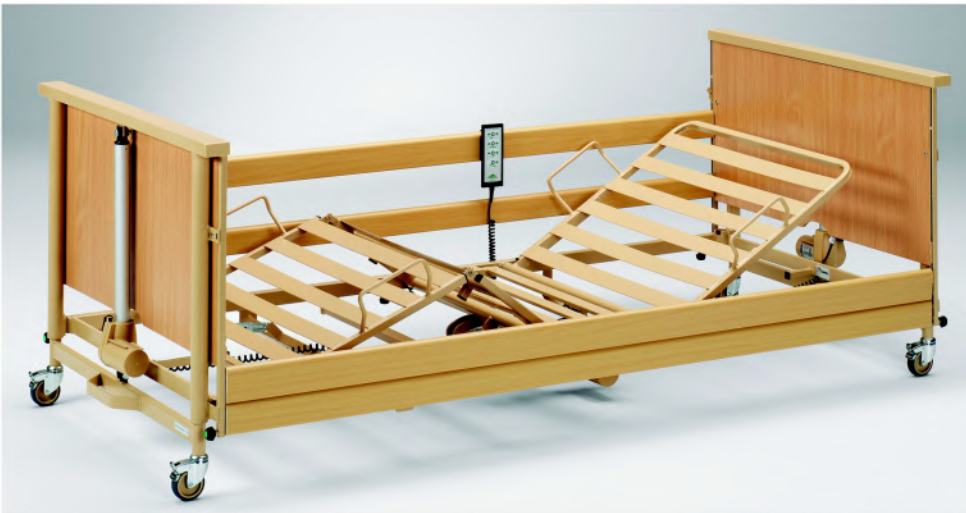

Australian Distributor

EVO CARE AUSTRALIA PTY LIMITED

A.B.N. 98 078 566 604

Trading as **EVO CARE** and **L&M EQUIPMENT**

P.O. Box 1144, Stafford Qld. 4053

Ph: 07 3355 8000 Fax: 07 3355 5043

Website: <http://www.evocare.com.au>

Email: sales@evocare.com.au

workshop@evocare.com.au

warehouse@evocare.com.au

accounts@evocare.com.au



Global-Mark.com.au®

Burmeier GmbH & Co. KG

(A company in the Stieglmeyer Group)


Pivitsheider Straße 270 • D - 32791 Lage/Lippe

Tel. : 0 52 32/98 41- 0 • Fax: 0 52 32/98 41- 41

Email: service-zentrum@burmeier.de




Status: 14.12.2012

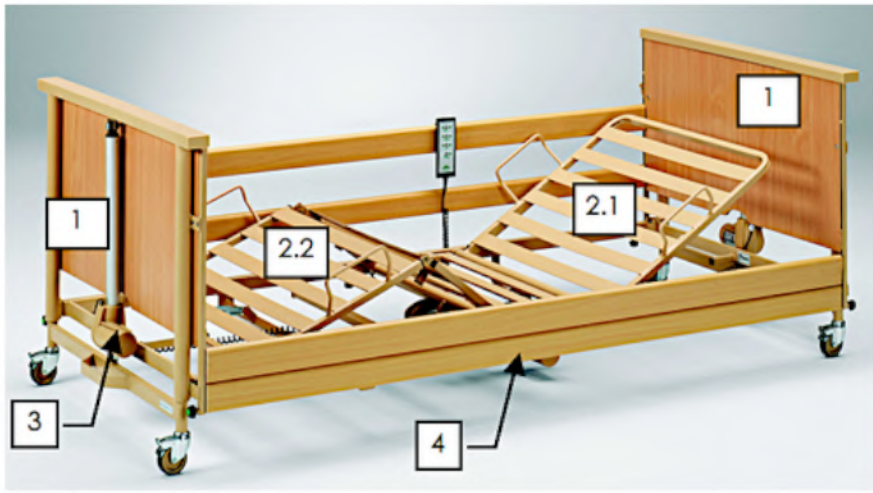



Dalil Low Entry Care Bed

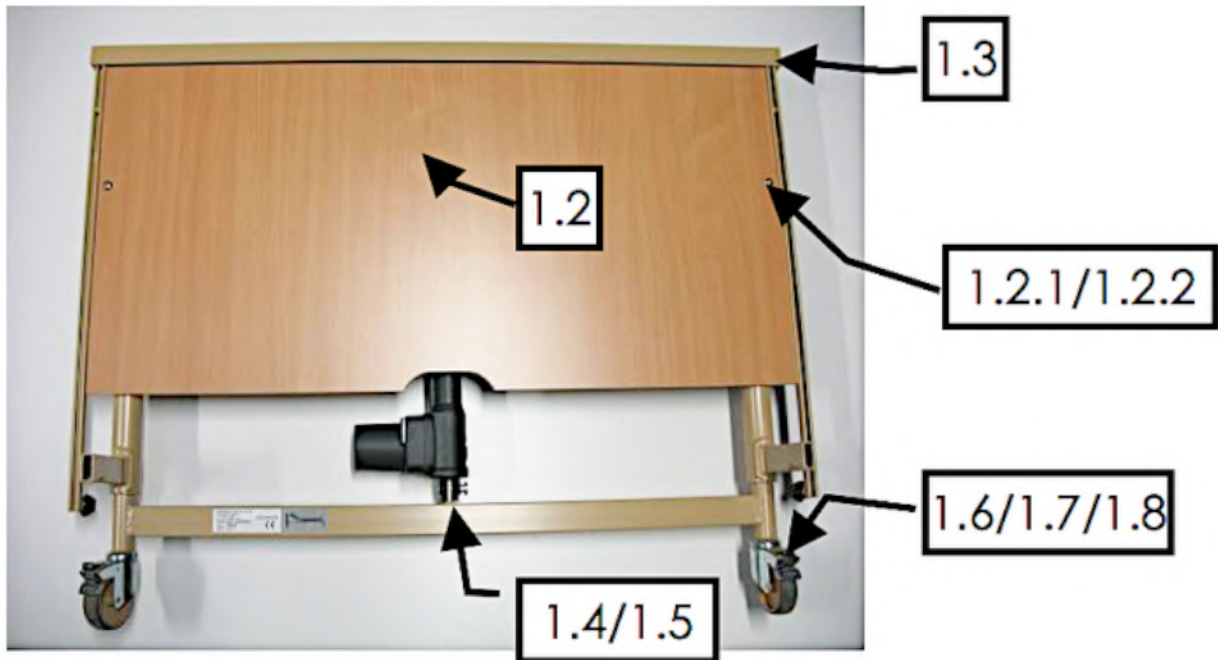1. Chassis ends



1.1 Chassis ends

1.1.1		chassis head section with lifting motor LA24 and wooden board	216292
1.1.2		chassis foot section with lifting motor LA24 and wooden board	216293

1.1 Chassis ends

1.2		wooden board	175271
1.3		castor Ø75mm	171192
1.3.1		screw M10 x 30mm	177829
1.3.2		Serrated lock washer Ø 10.5mm	209395
1.4		Tri-Grip bolt M8 x 18mm	152251
1.5		sleeved nut for wooden board, 13 x 8 x 10 x M6	106934
1.6		Bolt M6 x 12	166107


1.1 Chassis ends

1.7		plug for transverse spar 60 x 30mm	164825
1.8		Bolt for lifting motor $\varnothing 10$ x 35mm	182476
1.9		Locking washer for lifting motor $\varnothing 10$ mm	102555



2 mattress base

2 mattress base			
2		Metal slat mattress base (200 x 90cm) with motors	195317

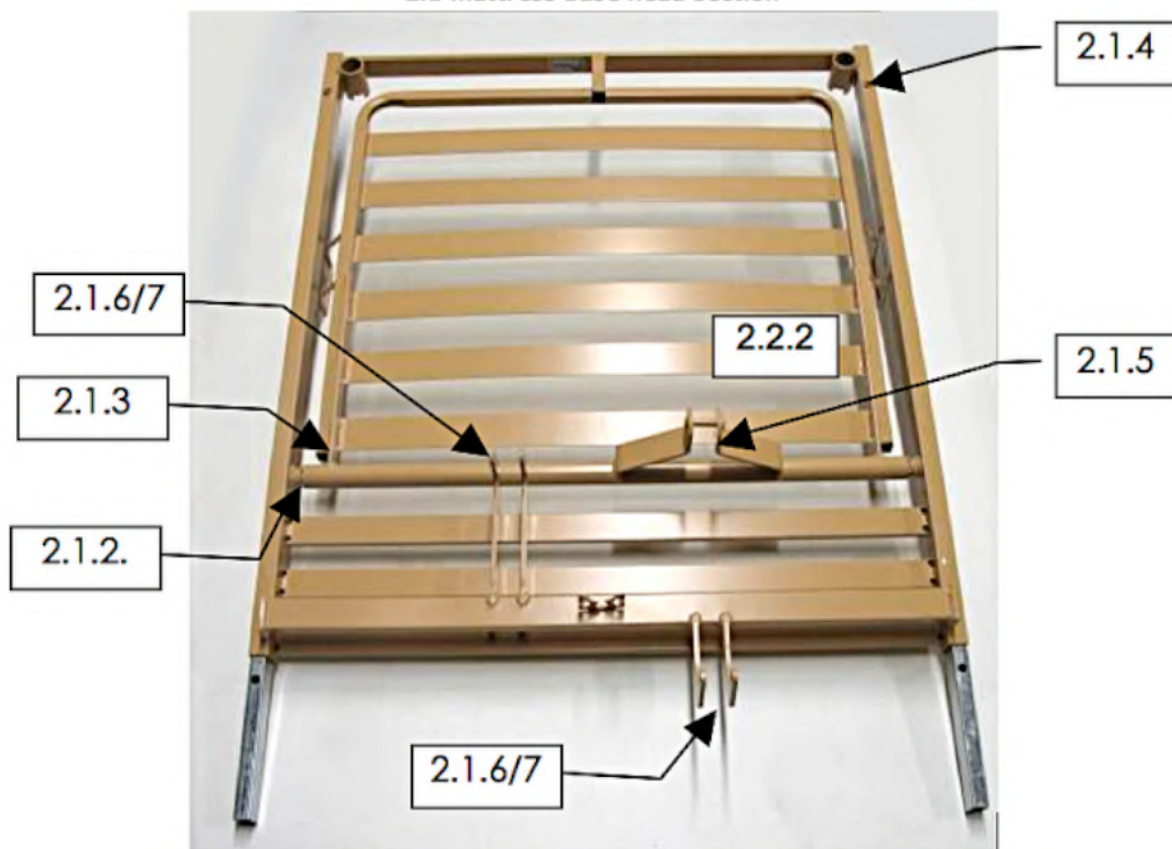2.1 mattress base head section




2.1 mattress base head section, metal slats

2.1		mattress base head section, metal slats, without motor	197664
2.1.1		back rest (inner frame) without motor, metal slats	209521

2.1 mattress base head section, metal slats

2.1.2		circlip, metal / size15 DIN6799	167385
2.1.3		Black square plug, plastic 20 x 20 x 3mm	100105
2.1.4		Tri-Grip bolt M8 x 18mm	152251
2.1.5		quick release bolt, back rest motor, Ø10 x 60mm	200395
2.1.6		collar bolt Ø10 x 55mm (for back rest / thigh rest motor)	200394
2.1.7		lock washer (open) Ø10mm (for back rest / thigh rest motor)	102555

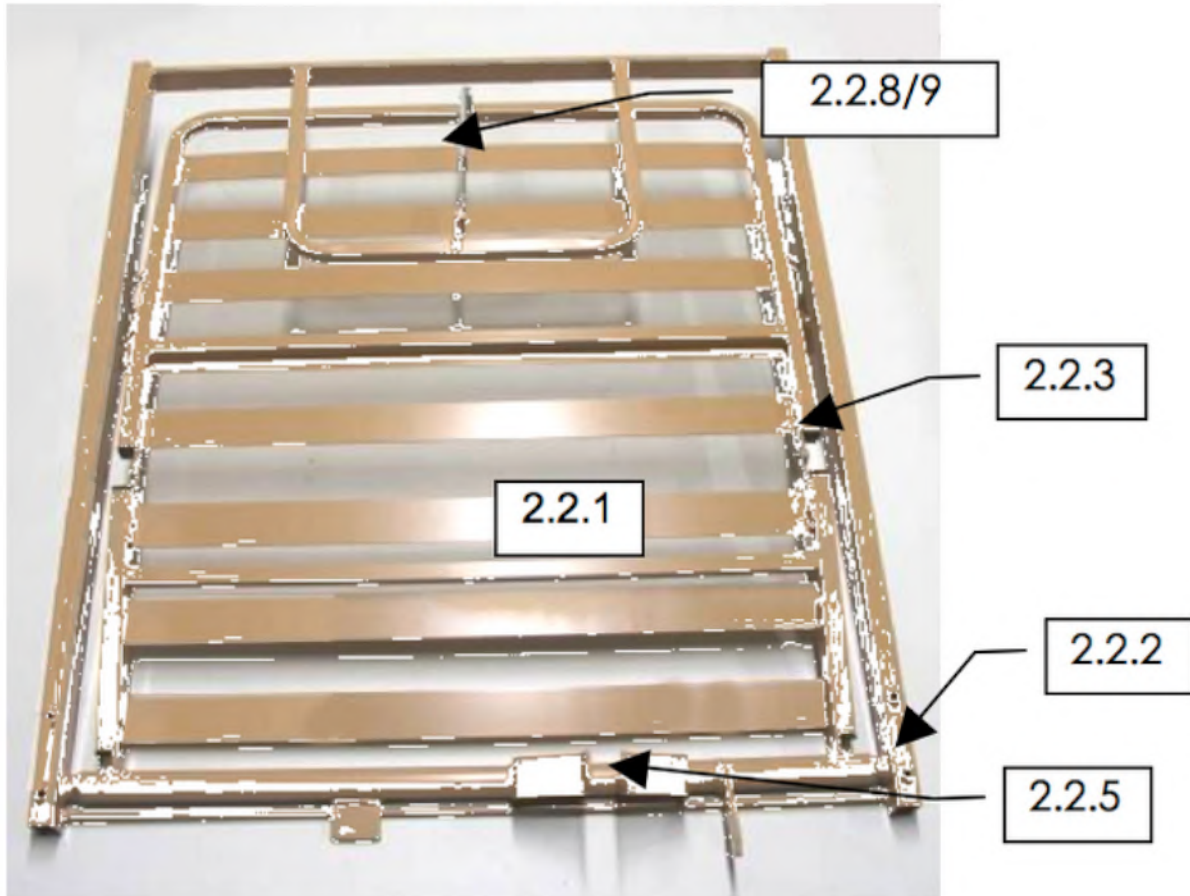

2.2 mattress base foot section






2.2 mattress base foot section




2.2 mattress base foot section			
2.2		Mattress base foot section without motor, metal slats	197666
2.2.1		thigh and lower leg rest (inner frame) without motor, metal slats	197663
2.2.2		circlip, metal / size15 DIN6799	167385

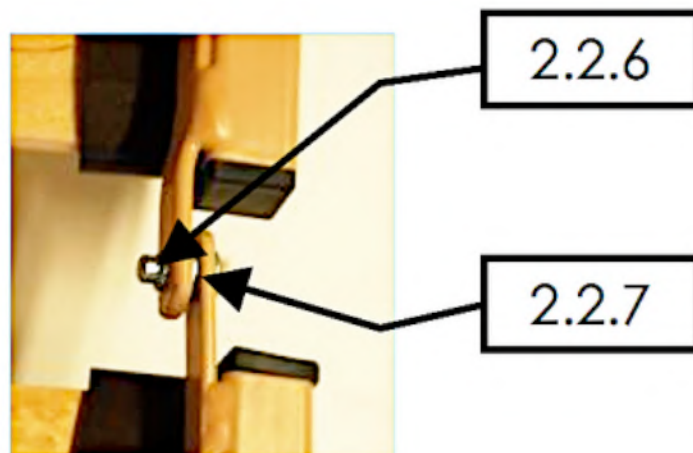2.2 mattress base foot section

2.2.3		Black square plug, plastic 20 x 20 x 3mm	100105
2.2.4		Tri-Grip bolt M8 x 18mm	152251
2.2.5		quick release bolt, thigh rest motor, Ø10 x 60mm	200395



2.2.6		blind rivet Ø6.4 x 14mm, (connection thigh / lower leg rest)	179334
2.2.7		washer, metal Ø6.4mm (connection thigh / lower leg rest)	106043

2.2 mattress base foot section			
2.2.8		ratchet, short	169827
2.2.9		lock washer Ø8mm (for ratchet)	106067
2.2.10		Self-adhesive white stripe 1000 x 18 x 1mm	192516




3. drive system

3. Drive System			
3.1		hand control with locking function	170704
3.1.1		hook for hand control	158919
3.1.2		locking key	167882
3.2		Switch Mode power supply SPMS4 (IP44)	202171



3. Drive System

3.3.1		back rest motor LA27 (previous model)	185070
3.3.2		back rest motor LA24 (follow-up model)	207578
3.4.1		thigh rest motor LA27 (previous model)	170701
3.4.2		thigh rest motor LA24 (follow-up model)	207579
3.5.1		lifting motor LA27 (previous model)	170702
3.5.2		lifting motor LA24 (follow-up model)	207577
3.6		cable harness 4-motors (24Volt)	170705

4. safety sides

4 safety sides			
4.1		assembly set for safety side guidance	188731
4.2		bracket buckle 330mm for safety side guide	191714
4.3		safety side, timber 2020 x 95.5mm	191245

5. lifting pole

5. lifting pole			
5		lifting pole	146602
5.1		lifting pole without grab handle	160802
5.2		grab handle	222225
5.3		lifting pole end cap, topas	182111

Accessories

Number	designation	article number
A	lifting pole with grab handle	146602
B	padded side rail cover for 2-bar side rail	225921
C	bed extension approx. 20cm, colour: topaz incl. 4 longer side rails 2225 x 95.5 x 33mm	195915 / 4 x 198167
D	reading lamp with fastening (claw) at lifting pole	175591 / 174231
E	reading lamp with adapter for the lifting pole sleeve	196221
F	storage and transport aid, colour: topas	196424
G	infusion holder (5200-28-50)	116354
H	wall deflector roll including bracket	206751
I	upholstery pad 100 x 200 x 12cm RG 40	242024
J	upholstery pad 100 x 220 x 12cm RG 40	242025
K	stand up aid	244029